
## **Canvas Wheelchair Cushion**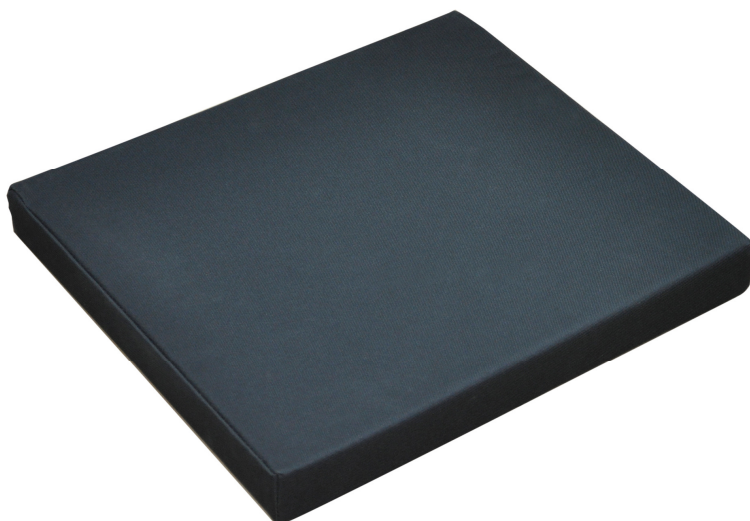

**VA126S**

## **Usage and Maintenance Instructions**

## INTRODUCTION

Thank you for deciding to purchase the Canvas Wheelchair Cushion from Aidapt. The Canvas Wheelchair Cushion is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

**NB. This equipment must be installed by a competent person.**

## BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the item.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## INTENDED USE

- The Canvas Wheelchair Cushion is designed to make your ride more comfortable when using a standard self-propelled or attendant-propelled wheelchair and can also provide extra support and comfort in cars and homes.
- Ensure that the Canvas Wheelchair Cushion is pushed back to the rear of the seat before use.
- Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or foam filling.
- While the Canvas Wheelchair Cushion is suitable for use on a wheelchair outdoors, do not leave the cushion open to the elements for long periods as the damp may cause damage to the cover or foam filling.

## CLEANING

Clean your Canvas Wheelchair Cushion using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not soak the cushion cover or filling.

After cleaning, leave the cushion to air/dry in a warm, dry place, but not in direct contact with a heat source.

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai Germany (Altai Europe Ltd)**
**Altai Nederland (Altai Europe Ltd)**