
Quick Suction Rail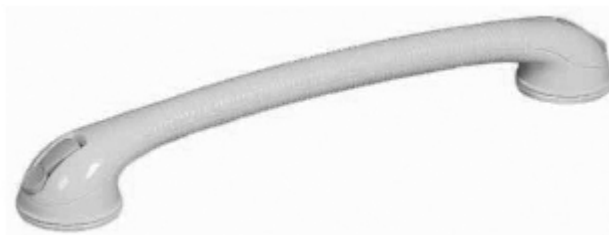

VY328

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Quick Suction Rail from Aidapt. The Quick Suction Rail is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

NB. This equipment must be installed by a competent person.

Maximum Load 30 kg.

Do not exceed the maximum load; doing so could put the user at risk.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

INTENDED USE

1. The Quick Rail is designed to assist your natural motion, as when getting in and out of a shower. It is not designed to hold or support your full body weight.
2. The Quick Rail can be used where permanent drilling and fixing is not possible.
3. Do not attach to moving surfaces.
- 3.4 The Quick Rail is designed for use only on smooth non-porous surfaces; tile, glass, porcelain, some plastics.
- 3.5 Not for use on painted, papered, wood, embossed surfaces.
- 3.6 Check the condition of the support surface before positioning the Quick Rail.
- 3.7 Support surfaces should be sound and free from defects.
- 3.8 Do not mount the Quick Rail where it will be in direct sunlight.
- 3.9 Check the condition of the suction cups. Do not use if damaged, cracked or split.
- 3.10 Do not attempt to mount the Quick Rail on mosaic or embossed tiles.
- 3.11 Check secure attachment before each use.

FITTING

1. Thoroughly clean the surface with a solvent cleaner or alcohol wipe and allow to dry.
2. Raise both levers; place the Quick Rail so that both suction cups lie firmly and completely on a smooth surface.
3. Do not place over joints or grout lines.
4. Press on one lever at one end, then the other. You should feel resistance as the suction cup tightens.

REMOVAL

1. Lift each of the two levers.
2. If suction remains, gently lift one edge with a finger.
3. Do not use a sharp instrument which may damage the suction cup.
4. Clean the Quick Rail from time to time with a non-abrasive cleaner.

LIABILITY WARNING

- The Quick Rail is designed for use only on smooth non-porous surfaces.
- Check the stability of the mounting surfaces before installation.
- Always check attachment before each use.

THE USER ASSUMES ALL RISK OF INJURY DUE TO USE

As the conditions in which the Quick Rail is used are beyond the control of the manufacturer or supplier, it is the obligation of the user to determine the correct and safe selection and conditions of use of the Quick Rail. To the extent that the law permits the supplier's liability is limited to the cost of repairing or replacing the products at its discretion, and does not extend to any loss or damage which may be caused as a consequence of misuse or improper or defective installation. In no event shall the supplier be liable for any other loss or damage, whether direct, indirect, foreseeable, consequential or special (including, but not limited to physical injuries or loss of use, revenue or profit) whether based on warranty, contract, tort (including negligence) or strict liability arising in connection with the use of the Quick Rail

CLEANING

Clean using a non-abrasive cleaner or mild detergent with a soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Always ensure that you wipe dry all equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk

Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)