## **Writing Grip for Pencils**

**VM965**

## **Usage and Maintenance Instructions**

## **INTRODUCTION**

Thank you for deciding to purchase the Pencil Grip from Aidapt. The Pencil Grip is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

## **BEFORE USE**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **INTENDED USE**

The Pencil Grip is an affordable, comfortable writing aid for those who have trouble gripping. The Pencil Grip is popular with users who suffer from arthritis, who often find gripping a normal pencil for writing to be difficult or painful.

## **INSTRUCTIONS FOR USE**

- The pencil grip can be used with any standard-sized pencil – whether it has a smooth, round or faceted circumference.
- Slip the pencil through the hole in the centre of the pencil grip, pushing the grip as far along the pencil as is required to reach a comfortable position for gripping the pencil to write.
- The Pencil Grip can be removed and replaced as necessary; holding the pencil grip firmly pull the pencil out, with small side to side movements if necessary.

## **CLEANING**

Clean your Pencil Grip using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai Germany (Altai Europe Ltd.)**