Writing Grip for Pens & Pencils

VM964

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Writing Grip from Aidapt. The Writing Grip is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Writing Grip slips over any standard size pen or pencil. The moulded finger grip offers a comfortable writing position, which is ideal for those with limited strength or grip in the hand, wrist or arm.

INSTRUCTIONS FOR USE

- The writing grip can be used with any standard-sized pen or pencil – whether it has a smooth, round or faceted circumference.
- Slip the pen or pencil through the hole in the centre of the writing grip, pushing the grip as far along the pen or pencil as is required to reach a comfortable position for gripping it to write.
- The Writing Grip can be removed and replaced as necessary; holding the writing grip firmly pull the pen or pencil out, with small side to side movements if necessary.

CLEANING

Clean your Writing Grip using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)