## **Lightweight Dressing Stick**

**VM950**

## **Usage and Maintenance Instructions**

## **INTRODUCTION**

Thank you for deciding to purchase the Lightweight Dressing Aid from Aidapt. The Dressing Aid is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

## **BEFORE USE**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **INTENDED USE**

A lightweight dressing aid featuring a smooth wooden shaft with a large C hook at one end and a specially designed push/pull hook at the other, the Dressing Stick is particularly popular with those who have limited mobility, or the use of only one hand.

## **INSTRUCTIONS FOR USE**

- The hook/pusher assists in putting on shirts, cardigans and jackets, pulling up and adjusting trousers or skirts, or removing socks
- The small hook on the opposite end aids in pulling zips and shoelace loops

## **CLEANING**

Clean your Dressing Stick using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you rinse then wipe dry the equipment after cleaning.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai Germany (Altai Europe Ltd.)**