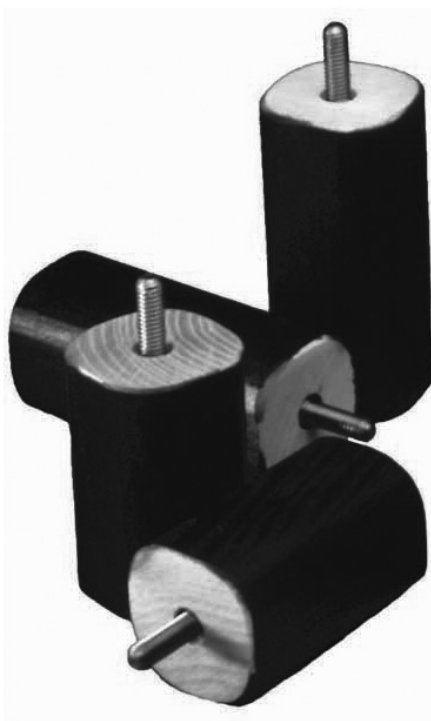
Screw-Fit Bed Raisers

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Screw-Fit Bed Raisers from Aidapt. The Screw-Fit Bed Raisers are manufactured from the finest materials available. When used correctly they are designed to give many years of reliable, trouble-free service.

NB. This equipment must be installed by a competent person.

VG824 Bed Raisers: 75 mm (Set of 4)

VG825 Bed Raisers: 100 mm (Set of 4)

Maximum User Weight: 380 kg (60 stone) including furniture.

Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

Screw-Fit Bed Raisers are available in two lengths 75 mm and 100 mm to raise the height of a bed fitted with wooden legs.

They are manufactured from hardwood and finished in black lacquer with strong steel fitting inserts.

FITTING YOUR SCREW-FIT BED RAISERS

NB. Two people may be required to fit the Screw-Fit Bed Raisers safely.

1. Remove original bed castors from their socket fittings.
2. Fit each bed raiser fully home into the fitting of the bed sockets.
3. The raiser must be thoroughly tightened. Ensure there is no gap between the top mating face of the bed raiser and the bed base.

NB: The base of the bed must be a flat surface.

4. Now replace the original castors into the bottom fitting of the Bed Raiser. The leg must be thoroughly tightened
5. Maximum user weight: 380 kg. (including weight of bed) per set of four raisers.

CLEANING

Clean your Bed Raisers using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. After cleaning, wipe the raisers with a dry, soft cloth if necessary. Do not leave in direct contact with a heat source.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

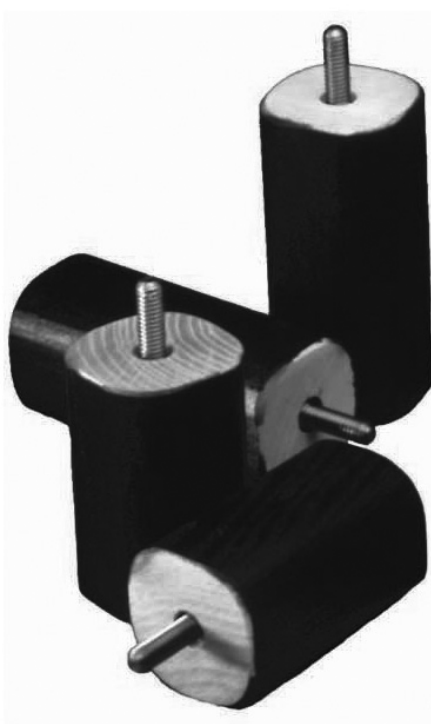
Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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**Altai Germany (Altai Europe Ltd.)