Deluxe Pressure Relief Orthopaedic Coccyx Cushion

VM974AB

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Deluxe Pressure Relief Orthopaedic Coccyx Cushion from Aidapt. The Deluxe Pressure Relief Orthopaedic Coccyx Cushion is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

INTENDED USE

The Deluxe Pressure Relief Orthopaedic Coccyx Cushion is designed to make your ride more comfortable when using a standard self-propelled or attendant-propelled wheelchair and can also provide extra support and comfort in cars and homes.

NB. Ensure that the Cushion is pushed back to the rear of the seat before use and that you tie the securing straps around the chair back or wheelchair frame if possible.

Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or foam filling.

While this product is suitable for use on a wheelchair outdoors, do not leave the cushion open to the elements for long periods as the damp may cause damage to the cover or foam filling.

CLEANING

Clean your Deluxe Pressure Relief Orthopaedic Coccyx Cushion using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not soak the foam inner. The cover can be removed and washed at a low heat with similar colours.

After cleaning, leave the cushion and cover to dry in a warm, dry place, but not in direct contact with a heat source.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk

Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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**Altai Germany (Altai Europe Ltd.), Reklamationen & Rücksendungen an: