



Scooter or Wheelchair Lap Straps



Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Scooter or Wheelchair Lap Strap from Aidapt. The Scooter or Wheelchair Lap Strap is manufactured from the finest materials available.

When used correctly it is designed to give many years of reliable, trouble-free service.

VA121 Lap Strap for Wheelchairs

VA121S Lap Strap for Wheelchairs or Scooters

NB. This equipment must be installed by a competent person.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the item.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

Before use ensure that the strap is threaded through the buckles and the snap fastener correctly, as in Fig. 1.



Fig. 1

INTENDED USE

Our safety lap straps use a snap lock fastener and are adjustable to fit. They are available in two styles;

The VA121 is secured around the backrest or armrest of the frame and is primarily for use on wheelchairs.

The VA121S passes entirely around the seat back and is mainly for use with scooters but can also be used with wheelchairs.

FITTING INSTRUCTIONS

VA121: Remove the buckle and clasp from the strap first and then loosen enough screws from the back panel of the wheelchair (enough to allow the strap to be passed between the back of the chair and the frame). Pass the free end of the strap around the back of the chair and thread through the metal slide ring, make sure to pull the strap tight. Make sure to tighten all loosened screws and replace the buckle and clasp on the strap. (See Fig. 2)

VA121S: Form a loop through the buckle at one end of the strap (see Fig. 3), pass this over the arm of the wheelchair and pull the loop tight (Fig. 4). Pass the strap around the back of the wheelchair and make another loop at the other side, again passing this over the arm.



Fig. 2



Fig. 3



Fig. 4

CLEANING

Clean your Scooter or Wheelchair Lap Strap using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner.

After cleaning, leave the strap somewhere warm and dry to air. Do not leave in direct contact with a heat source.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk