Crutch Bag

VA132SB

Fitting and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Scooter or Wheelchair Crutch Bag from Aidapt. The Scooter or Wheelchair Crutch Bag is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

INTENDED USE

Scooter or Wheelchair Crutch Bags are suitable for use with most standard sized wheelchairs and scooters.

The Scooter or Wheelchair Crutch Bag is designed to provide useful, hands-free storage for wheelchair and scooter users and Carers, with specially-designed pockets for storing crutches or walking sticks.

FITTING INSTRUCTIONS

The Scooter or Wheelchair Crutch Bag attaches to the rear of the wheelchair with strong, adjustable webbing straps or hanging loops.

To use with Wheelchairs simply hook a looped strap onto each of the handles to suspend the bag from the back of the wheelchair.

To use with Scooters use the adjustable straps to secure the bag to the seat of the scooter; squeeze the buckles to unfasten, adjust the length as required and push the buckle until it clicks to fasten. Straps can be secured straight or crossed over for the best fit.

When carrying crutches or walking sticks adjust the crutch or walking stick to the minimum height before inserting into the interior pockets either side of the main storage area. Use the hook and loop straps to secure them snugly.

CLEANING

Clean your Scooter or Wheelchair Crutch Bag using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner.

After cleaning, leave the bag somewhere warm and dry to air. Do not leave in direct contact with a heat source.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Nederland (Altai Europe Ltd)