Memory Foam Leg Pillow

VM936D

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Memory Foam Leg Pillow from Aidapt. The Memory Foam Leg Pillow is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

INTENDED USE

The ergonomically-designed Memory Foam Leg Pillow can help to provide support to stiff or sore legs or painful knees; it is ideal for use in a bed or flat surface and is constructed from memory foam with a washable, removable velour cover.

NB. Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or foam filling.

The Memory Foam Leg Pillow has an adjustable, elasticated strap (with hook and loop fastening) attached to the cover, which you can wrap around your leg to keep the cushion secure in the most comfortable position.

CLEANING

Clean your Memory Foam Leg Pillow using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner or bleach-based cleaning product as this may damage the foam or cover. Do not soak the foam inner. The cover can be removed and washed at a low heat with similar colours.

After cleaning, leave the cushion and cover to dry in a warm, dry place, but not in direct contact with a heat source.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk

Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai France (Altai Europe Ltd)
Altai Germany (Altai Europe Ltd.)