



Inflatable Pressure Relief Ring Cushion



VM934B/VM934BB (With Pump)

Usage and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

INTENDED USE

The Inflatable Pressure Relief Ring Cushion from Aidapt reduces the pressure when sitting. It also helps to distribute weight evenly and makes it easier for the user to maintain the correct seated position.

Inflate the cushion by mouth or using a pump (such as a hand pump for airbeds or similar. Pump not supplied.).

NB. Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or inflatable inner.

NB. Never use a compressor to inflate the cushion.

NB. When inflating this item by mouth you may need to bite down on the base of the valve whilst blowing so as to open up the airway.

CLEANING

Clean your Inflatable Cushion using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. After cleaning, wipe the Inflatable Cushion with a dry cloth before leaving in a warm, dry place, but not in direct contact with a heat source.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

NB. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9
3EX**

**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web:
www.aidapt.com**

Email: sales@aidapt.co.uk