



## Lightweight Suitcase Ramp



### **Fitting and Maintenance Instructions**

**Please read this manual before operation for maintenance instructions and safe usage of these ramps.**



## **MAINTENANCE & SAFETY**

- Regularly check your ramps for worn, loose or damaged parts – If you find any of these to be damaged you must not use your ramps and you must contact your dealer immediately.
- Always use both ramps, and ensure they are level and parallel to each other.
- Ensure ramps are fully extended and locked in position.
- Do not attempt to use ramps on gradients above 1 in 12. Please also check the gradient your wheelchair or scooter is capable of.
- Do not exceed the weight capacity of the ramps.
- Only allow one person/ mobility vehicle to use the ramps at once.
- The surface of the ramps may become slippery in icy or wet conditions. Please exercise caution in these conditions.
- Keep your ramps clean of any dirt or debris as this could interfere with its sliding motion.
- Always use the ramps in line with the wheelchairs/scooter manufacturer's guidelines; NEVER use it at a steeper gradient than recommended.
- Do not modify or add to your ramp as it could void any warranty claims.
- Do not use the ramp for anything other than its original intended purpose.
- Only clean your ramps with mild detergent.
- Use caution when adjusting the length of your ramps and avoid any pinch points.

## **CLEANING**

Clean your Ramps using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner.

After cleaning, leave the Ramp somewhere warm and dry to air or wipe over with a dry, soft cloth. Do not leave in direct contact with a heat source.

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**