Hygienic Bottom Wiper

VM971D

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Hygienic Bottom Wiper from Aidapt. Aidapt Personal Toileting Aids are manufactured from the finest materials and components available. When used correctly they are designed to give many years of reliable trouble-free service.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

HOW TO USE

- Hold toilet tissue or moist wipe next to the indented section at the end of Hygienic Bottom Wiper and depress the button at the end of the ergonomically-designed handle to grasp the tissue or moist wipe.
- After use, release the button to release the grip on the toilet tissue or moist wipe and deposit it into the toilet or commode bucket. NB. Please check and comply with the manufacturer's recommended method for disposing of moist wipes.

CLEANING

Clean your Personal Toileting Aid using a suitable disinfectant. Abrasive cleaners and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)
Altai Nederland (Altai Europe Ltd)