Weekly Pill Dispensers

VM930AB/VM931AB

Usage and Maintenance Instructions

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The Weekly Pill Dispenser is for use as a general pill sorter or a pill dispenser during holidays. It has 7 clearly labeled compartments; 1 for each day of the week.

USING YOUR PILL DISPENSER

- To open the daily tablet containers, locate the button at the front of the compartment and press.
- To close, push down on the lid firmly until you feel or hear a slight click as the lid clicks into place.

NB. Do not overfill the compartments as this may cause damage to the pills or prevent the lid from closing properly.

CLEANING

Clean your Weekly Pill Dispenser using a non-abrasive cleaner or mild detergent with a soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Always ensure that you wipe dry all equipment.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai France (Altai Europe Ltd)
Altai Germany (Altai Europe Ltd.)