



## VA116SS Scooter Full Weather Protection Cover

### Usage and Maintenance Instructions

#### **INTRODUCTION**

Thank you for deciding to purchase the Scooter Full Weather Protection Cover from Aidapt.

**NB. A RISK ASSESSMENT SHOULD BE UNDERTAKEN AS TO THE SUITABILITY OF THE PRODUCT FOR THE PARTICULAR USAGE.**

#### **BEFORE USE**

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage if you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

#### **INTENDED USE**

This Scooter Full Weather Protection Cover from Aidapt provides protection from the elements. It can be put on or taken off with ease and features a snug fitting hood.

#### **FITTING INSTRUCTIONS**

The Cover can be easily slipped on and off then adjusted as necessary:

- Unzip the neck opening fully and put the poncho on over the head
- The neck opening can be zipped up or down as desired and the hood can be raised and tied

**NB. Ensure skirt of product is clear of scooter wheels to avoid entrapment**

**NB. Keep Scooter Cover away from any moving parts of the scooter**

## **CLEANING**

Clean your Scooter Cover using a mild detergent with a slightly damp, soft cloth only. Abrasive cleaners and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. The Scooter Cover can be cleaned in a washing machine at a low temperature, ideally on a wool or delicate setting. After cleaning, hang the scooter poncho somewhere warm and dry to air. Do not leave in direct contact with a heat source and do not put in a tumble dryer.

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. **If in any doubt, please do not issue or use, immediately contact your supplier for service support.**

## **IMPORTANT INFORMATION**

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.com](http://www.aidapt.com) Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**