

UK  
CA

CE

aidapt

**VR205  
Ashford Toilet Frame**



**This file is available to view and download as a PDF at [www.aidapt.co.uk](http://www.aidapt.co.uk). Sight impaired customers can use a free PDF Reader (such as [adobe.com/reader](http://adobe.com/reader)) to zoom in and increase the text size for improved readability.**

**Fixing and Maintenance Instructions**

**NB. This equipment must be installed by a competent person.**

VR205 Ashford Toilet Frame

**Weight Limit: 190kg (30 stone)**

**Do not exceed the weight limit stated- doing so could put the user at risk.**

## BEFORE USE

Please examine your toilet frame for any visible damage before use. If you see any damage or suspect a fault, please contact your supplier.

## CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your toilet frame using a non-abrasive cleaner or mild detergent with a soft cloth, avoid using limescale remover. Abrasive cleaners e.g. AJAX and or abrasive cleaning pads may seriously damage your product beyond repair.

Certain products have clip on/clip off backs and/or seats; you can easily remove these if required, clean with a mild detergent and soft cloth in a warm water solution. Do not exceed 50°C when cleaning the seat cover and/or back as this will permanently damage the plastic.

## Regular daily cleaning will reduce the build up of limescale on your product


Under no circumstances use any abrasive cleaners or materials on any of the components. Also check that the cleaner you intend to use is suitable for ABS, Vinyl, Polymernr Plastics and Thremo Plastic Rubbers and Polyurethane Rubbers.

## ADJUSTMENT TO EXTENSION LEGS

Remove the 'E' clip from the extension leg and extend all four legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and is fitting neatly and snug around the outside leg (see fig. 2)

**Never install the extension legs unevenly or slanting, this will compromise the safety of the user.**

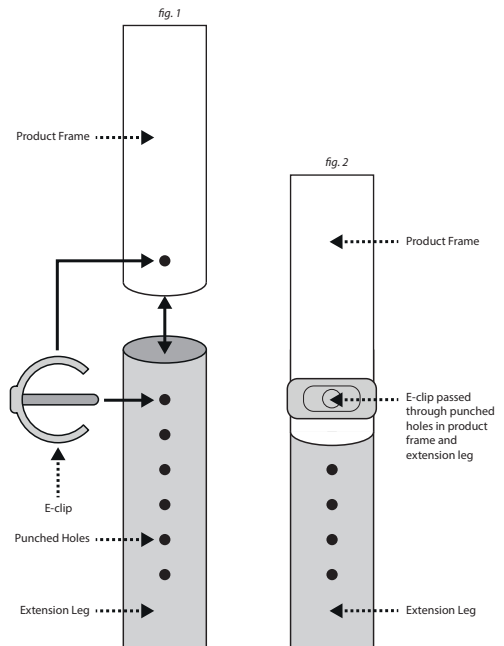

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- PAT test for electrical items
- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components
- Also check all upholstery for security, splits, etc.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 •**  
**Web: [www.aidapt.com](http://www.aidapt.com) • Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**