



**GH104**

## **Folding Adjustable Wooden Bed Tray**



## **Fixing and Maintenance Instructions**

### **INTRODUCTION**

Thank you for deciding to purchase this bed tray from Sutton Home & Garden. When used correctly it is designed to give many years of reliable, trouble-free service.

### **BEFORE USE**

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

### **INTENDED USE**

An attractive and functional bed/lap tray with folding legs and an adjustable laminated surface for easy cleaning.

## **SAFETY**

- Do not allow children to play with this item.
- Take care when handling the Wooden Bed Tray and be sure to hold it firmly at either end.
- Extra care should be taken when unfolding and folding the legs of the Wooden Bed Tray and changing the angle of the table top to avoid trapping fingers.
- It is not recommended that your tray be used for serving or carrying hot drinks or food.

***Do not overload the Wooden Bed Tray as this may damage the item or cause injury to the user***

## **INSTRUCTIONS FOR USE**

- Unfold the wooden legs on the right and left hand side by pulling them lightly down and out.
- Adjust the table top by lifting it and placing the bar beneath it into one of the grooves to set it to the correct height.
- The Wooden Bed Tray can now be placed over your lap, with the legs resting on the bed either side of you.
- To use the Wooden Bed Tray flat or store it away, simply fold both sets of legs under the body of the tray and lower the table top.

## **CLEANING**

Clean your Bed Tray using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners and/or abrasive cleaning pads may damage the product beyond repair and should not be used.

Always ensure that you wipe dry the equipment after cleaning.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

***If in any doubt, please do not issue or use, but immediately contact your supplier for service support.***

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **SERVICE WARRANTY**

Electrovision guarantees the product free from defects in materials and workmanship for a period of one year.

Should this unit be operated under conditions other than those recommended, at voltages other than the voltage indicated on the unit, or any attempts made to service or modify the unit, then the warranty will be rendered void.

The product you buy may sometimes differ slightly from illustrations. This warranty is in addition to, and does not affect, your statutory rights.

Electrovision Ltd., Lancots Lane, Sutton Oak, St. Helens, Merseyside WA9 3EX

website: [www.electrovision.co.uk](http://www.electrovision.co.uk)