



## **Bunion Splints**



**VM949BN**

**Usage and Maintenance Instructions**

## **INTRODUCTION**

Thank you for purchasing Bunion Splints from Aidapt.

## **BEFORE USE**

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **INTENDED USE**

The Bunion Splint is a corrective brace that can help provide support and protection to your big toe. The bunion splint can supply gentle pressure on your big toe to help realign the formation of bunions. It can help reduce pain and discomfort, as well as reduce pressure and inflammation of bunions. The splint is made from lightweight and breathable material that is comfortable for the user and makes it easy to clean. The splint is fully adjustable which can fit most individuals.

If you experience any pain or discomfort then cease using. If in doubt, consult a chiropodist or similar expert.

## **Instructions for Use**

1. Ensure the splint is securely fastened to your foot. Use the strap to adjust to your foot size.
2. Ensure your big toe is aligned and fastened to the splint with the toe strap. Use the strap to adjust to your toe size.

## **CLEANING**

Clean your Bunion Splints using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners may damage the product beyond repair and should not be used. Always ensure that you wipe the equipment dry after cleaning.

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.com](http://www.aidapt.com)**  
**Email: • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**