



## **Bed Rope Ladder**



**VM937**

## **Usage and Maintenance Instructions**

## **INTRODUCTION**

Thank you for purchasing the Bed Rope Ladder from Aidapt.

### **IMPORTANT**

**N.B This equipment must be installed by a competent person.**

**Do not exceed the maximum user weight of 160 kg (25 ST)**

## **BEFORE USE**

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## **INTENDED USE**

The Bed Rope Ladder is a simple bed aid, which allows you to independently pull yourself up to a desired seated position in bed.

## **Instructions for Use**

1. Attach the Bed Rope Ladder to the foot of the bed by tying the loose ends of the rope around the bed legs.
2. Ensure the rope ladder is securely attached to the bed before use.

## **CLEANING**

Clean your Bed Rope Ladder using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners may damage the product beyond repair and should not be used. Always ensure that you wipe the equipment dry after cleaning.

## **CARE & MAINTENANCE**

Please make a safety check of the product at regular intervals or if you have any concerns.

## **REISSUE**

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## **IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.com](http://www.aidapt.com)**  
**Email: • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**