

CEUK
CA

aidapt

Overbed Table



VG832AA

Usage and Maintenance Instructions

INTRODUCTION

Thank you for purchasing the Overbed Table from Aidapt.

BEFORE USE


- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

- The overbed table is both height and angle adjustable.

CONTENTS


- Wooden Finished Table-Top **(a)**
- Chrome L-Shaped Leg Brackets x2 **(b)**
- L Shaped Tube with Thumbwheel x2 **(c)**
- C Shaped Tube with Thumbwheels **(d)**

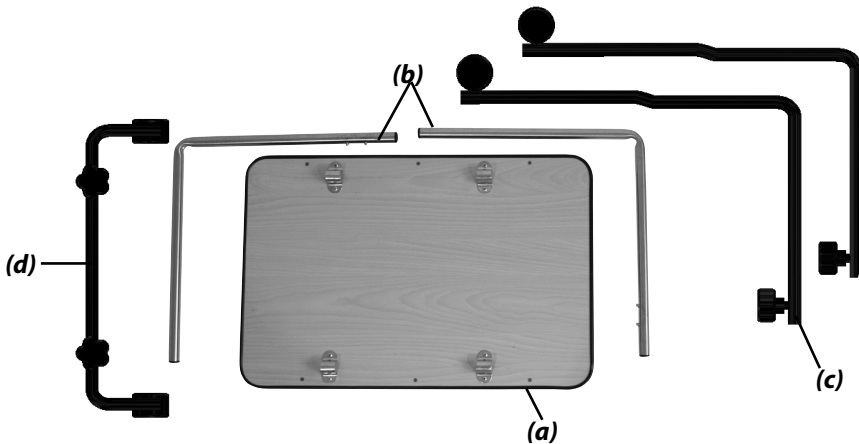


ASSEMBLY INSTRUCTIONS

1. Carefully unpack your Overbed Table and examine all parts for any damaged or, loose fits. Should you find any fault, please return your table to your supplier for exchange.
6. Slide L shaped tube **(c)** over chrome L-Shaped brackets **(b)** to desired height. Ensure they are both the same height. **(fig. 5)**


2. After inspection, place the wooden finished table-top face **(a)** down onto a soft surface to avoid scratches.
3. If required, insert the chrome leg brackets **(b)** into clamps on the wooden base **(a)**. **(fig. 1)**

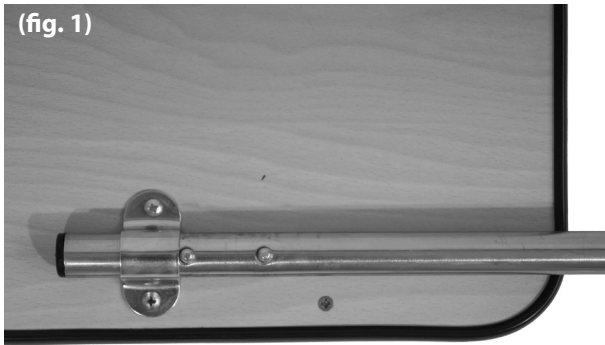


4. Depress click release pins. **(fig. 2-3)**

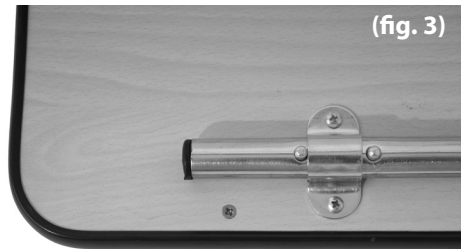

5. Attach C shaped tube **(d)** to L shaped tube **(c)** using the thumbwheels. **(fig 4)**

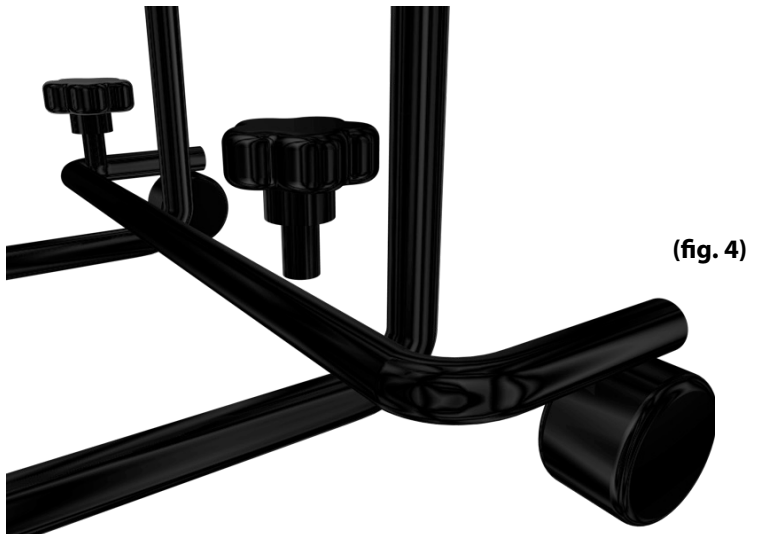

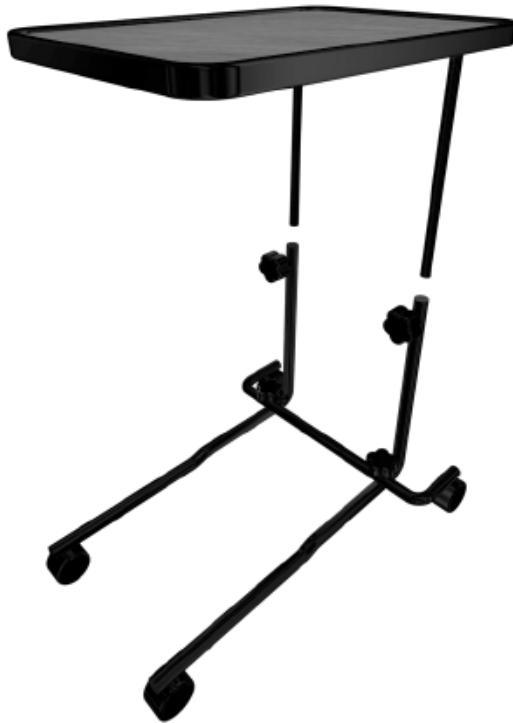


(fig. 5)

6. Slide L shaped tube **(c)** over end of chrome L-Shaped tube **(b)** making sure it is the end of the tube **(b)** with no click release pins (The ends with the click release plus fit under the table top)

Ensure they are both the same height. **(fig. 5)**

7. Tighten all thumbwheels by turning clockwise before use. **(fig. 6)**

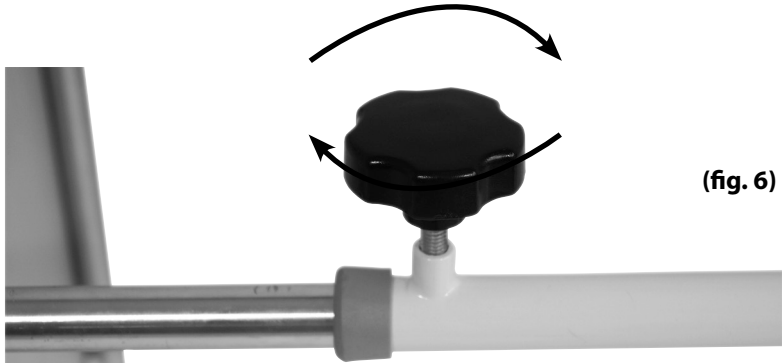

(fig. 6)

OPERATION INSTRUCTIONS

- To adjust the height and/or angle of the table, first loosen the two short thumbwheels by turning them anticlockwise. Adjust the table to suit and secure by turning thumbwheels clockwise.

NB. Ensure thumbwheels are adequately tightened before use.

NB. Ensure there is nothing on the table when adjusting

- **The table is not load bearing and should not be used to support objects over 15kgs (33lbs).**
- **Do not allow people to lean on the table.**
- **As the table is angle adjustable, please be careful with hot items to ensure they can not slip off the table.**

CLEANING

Clean your Overbed Table using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Using abrasive cleaning materials, very hot water or scrubbing the table top may result in damage.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product. Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/ use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 Fax: +44 (0) 1744 745 001

Web: www.aidapt.com

Email: sales@aidapt.co.uk