Height Adjustable Trolleys

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Height Adjustable Trolley from Aidapt.

The Trolley is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

IMPORTANT!

N.B. This equipment must be installed by a competent person.

Please Note: Always consult with the prescriber/Physiotherapist if you are in any doubt that this product is suitable for your use.

VG799 Height Adjustable Strolley Trolley
VG800 Height Adjustable Wingmore Trolley
VG801 Height Adjustable Meopham Trolley

Maximum Weight on Tray: 15 kg

Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the item.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

HEIGHT ADJUSTMENT

Your trolley can be adjusted in height if required.

- To adjust the height remove the lower 'E' clips from the extension legs and pull each leg to the desired height. (See fig. 1.)
- Ensure that all four extension legs are evenly adjusted.
- Re-insert all 'E' clips facing the line of travel also ensuring that the 'E' clips have passed through the product frame and extension leg and are secure. (See fig 2.)
- Never adjust the legs unevenly; this is dangerous and can seriously compromise the safety of the user.

Check that the trolley is sitting evenly on the floor and there is no excessive rocking or the trolley is tilting to one side. If this is the case, please do not use the trolley, but repeat the above steps to correct the problem or contact your supplier for service support.

CLEANING

Wingmore and Strolley Trolleys:

The polypropylene clip on trays can be removed for cleaning. To remove, place your finger and thumb on the underside of the U clip over inverted U section, push upwards and remove.

The tray can be cleaned with either a mild detergent or disinfectant, or alternatively pressure washed or autoclaved at a temperature of up to 100°C.

Meopham Trolley:

The MFC teak effect trays can be removed from the trolley prior to cleaning, to do so, firstly turn the trolley upside down and remove the four retaining screws holding the tray tops in place and then lift the tray(s) out of position.

Clean the MFC trays with a damp cloth, using a mild detergent or disinfectant solution. If you remove the tray, ensure that you reattach with new screws and ensure that the tray is secure. Do not saturate the trays with water, as this will damage the trays beyond repair. It is advisable to remove the trays from the trolleys before cleaning.

To refit the trays, simply push down on the inverted U sections ensuring that the tray is in line with the trolley frame. Check that all the engaging edges are sitting neatly, evenly and securely to the frame.

CLEANING THE CASTORS AND WHEELS

Aidapt trolleys are supplied with either polypropylene bodied castors or steel bodied castors.

The polypropylene castors or steel bodied castors can be removed from the frame by removing the 'E' clip retaining the extension leg. Once done, wash the castor in a bucket of mild detergent or disinfectant.

However ensure that you thoroughly dry the castor and extension leg prior to reassembly and finish by applying a water-dispersing agent such as WD40 over the castor wheel and body prior to reassembly. Following cleaning, please repeat all steps detailed in section 2, adjustment of leg with castor attached.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.


REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/bolts/castors
- Other screw in/bolt in/push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

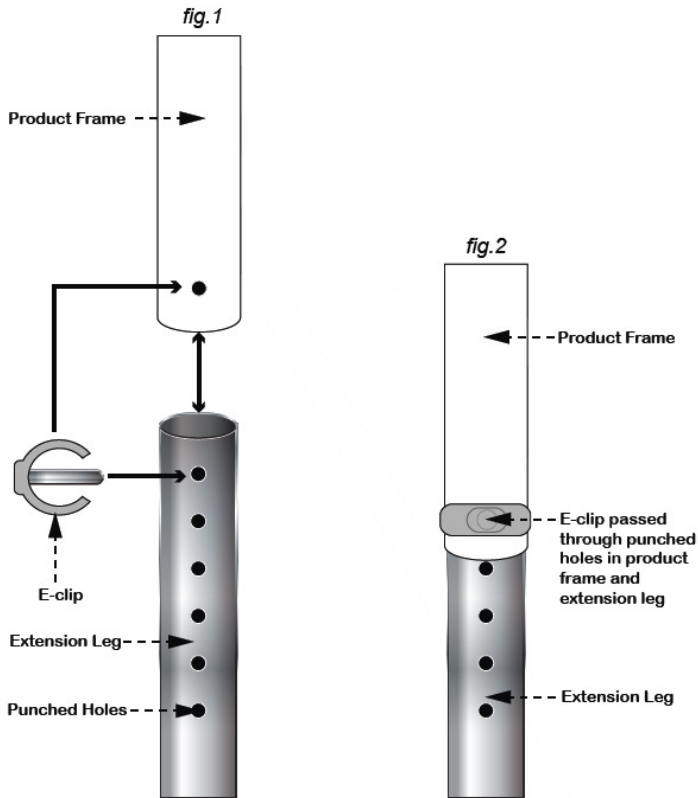IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Holland (Altai Europe Ltd)