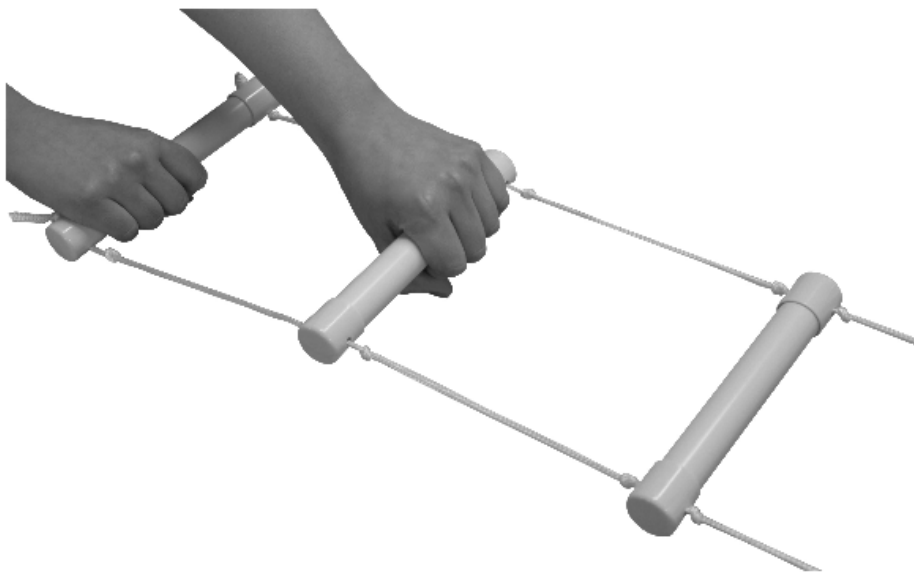
Rope Ladder Bed Hoist

VM937

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Rope Ladder Bed Hoist from Aidapt. The Rope Ladder Bed Hoist is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

NB. This equipment must be installed by a competent person and a risk assessment undertaken as to the suitability of the Rope Ladder Bed Hoist for the particular user and user's bed.

Max. User Weight: 100 kg (15 st.)

NB.: Do not exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

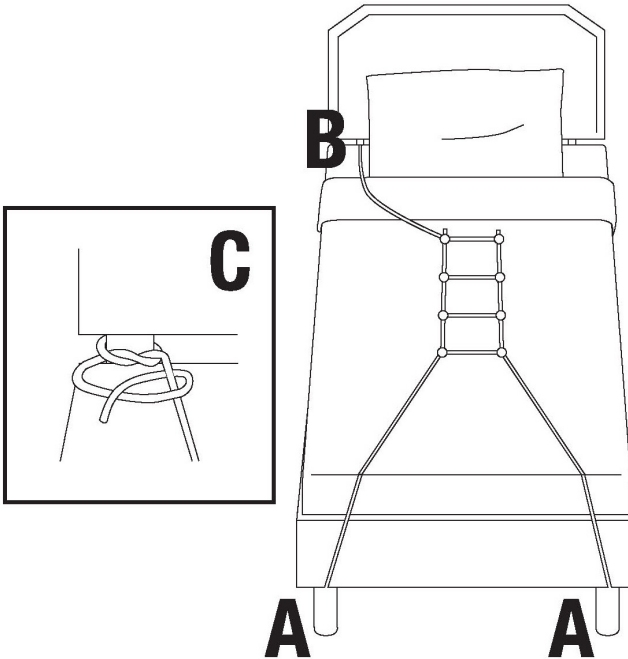
INTENDED USE

The Rope Ladder Bed Hoist helps the user to rise from a lying position to a sitting position. For use on beds that have legs.

INSTRUCTIONS FOR USE

Using the Rope Ladder Bed Hoist is simple, but requires a certain amount of dexterity:

- Attach the rope by means of a knot around the bottom legs of the bed (A) as shown in the diagram. The user can now pull him/herself up by use of the ladder.
- A string (not supplied) can be attached to the ladder then secured to the top of the bed so that the ladder is always ready for use. (See B on diagram)

CLEANING

Clean your Rope Ladder Bed Hoist using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used.

Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

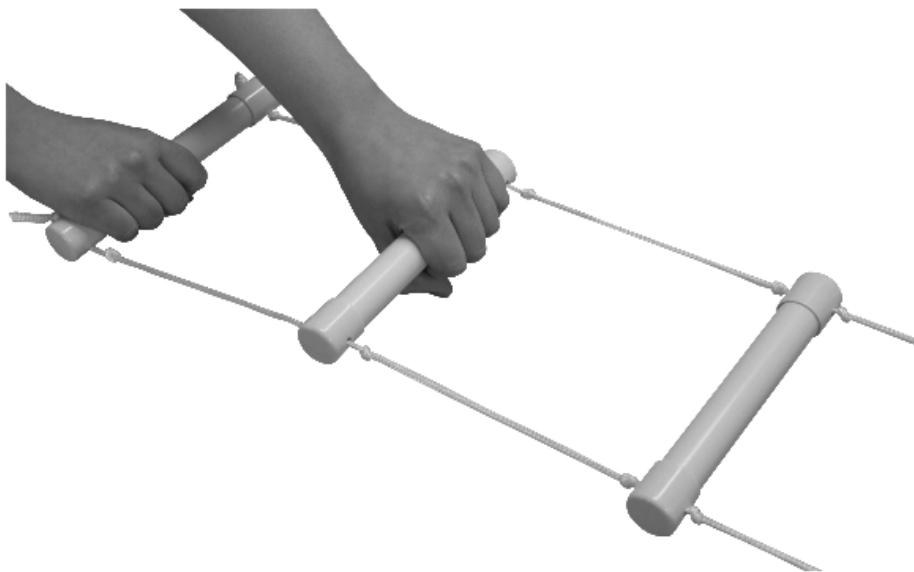
Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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**Altai Germany (Altai Europe Ltd.)