

AIDAPT BATHROOMS LTD.

Lancots Lane
Sutton Oak
St. Helens
WA9 3EX

VG305..... Pack of 50

VG305R Pack of 25

VG305S Pack of 10



Fluid Resistant Disposable Surgical Face Mask Type II R EN14683 certified earloop

Premium quality fluid resistant disposable medical surgical face masks can reduce the risk of contamination, preventing droplets being expelled into the environment by the wearer, exposure to contaminants and germs from others, like sneezing and coughing. Disposable surgical face masks help prevent large particles expelled by the wearer from reaching those in close proximity.

Important note: These masks have different layer materials and thickness to correctly reach Type II R Mask (EN14683) certification.


Our face masks are ideal for everyday use by healthcare professionals, care homes, medical centres and the general public to help protect and reduce the risk of transmission of infective agents, they are easy and comfortable to wear, thanks to their soft, elasticated latex free ear loops and lightweight design. These masks are 3 ply breathable with an integrated nose clip, the masks have been manufactured with different layer materials and thickness to ensure full compliance with type II R EN14683 certification.

Odourless and hypoallergenic 1 size fits all ensuring you are protected with the comfortable fit and filtering system with these face masks achieving a (BFE) exceeding 98% in certified tests.

The masks are not individually wrapped. Please note, due to the hygienic and contamination control properties of this product it is non-returnable.



Multi-layered protection against harmful substances

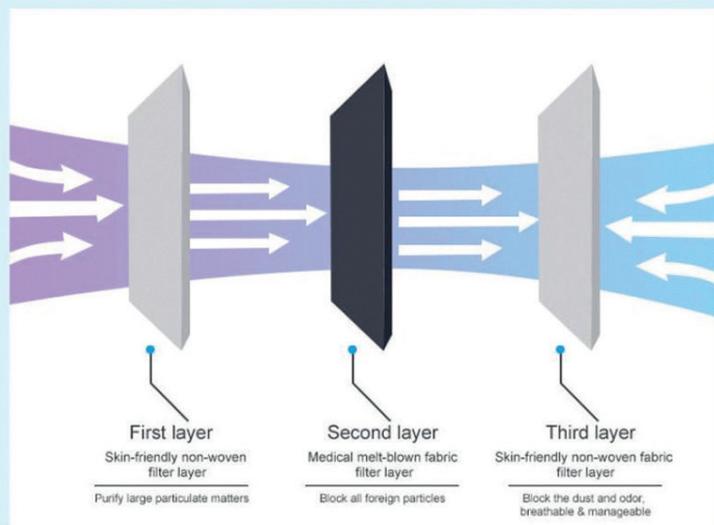

Instructions:

Wearing the disposable surgical face mask should be done in conjunction with regular hand washing.

Wash hands thoroughly before putting on a mask, and again before and after taking one off. The coloured side of the mask faces outwards with the nose strip and thicker white band at the top.

Firstly, put the mask over your mouth, tucking the ear loops over and behind your ears. The loops should hold the mask firmly on your face.

Then, you should hold the top and bottom of the mask and unfold it so that it covers your nose, mouth and chin.

Important note, the folds should go downwards for the outer surface! Wearing the mask upside down could result in the mask being a container for droplets, bacteria and viruses.

Finally, mould the nose strip to your nose and ensure the mask is not excessively loose.

The disposable surgical face mask is designed for single use only and should be disposed of after one day of use.

Store in a cool dry place out of direct sunlight. Keep away from sources of ozone or ignition.

Features:

- Integrated nose bridge moulds to your face to fit perfectly
- Latex-free elastic band
- Universal size fits all
- Easy and comfortable to wear
- Light blue colour.
- Non sterile

Specification:

- Complies with BS EN 14683 standard Type II R certification
- CE Compliant
- REACH EC NO 1907/2006
- Quality Management ISO 13485 Certified
- Bacterial filtration efficiency EN14683: 2019 Annex B (BFE) exceeds $\geq 98\%$
- Breathability: EN14683:2019 Annex C 26.7 Pa/cm²
- Splash resistance ≥ 16 kpa. ISO 22609:2004
- Microbial cleanliness: ENISO11737-2018 <20 cfu/g
- Biocompatibility: EN ISO10993
- 3 ply filler construction, type II-filtration exceeds 98% (meltblown filler)
- Outer layer: PP Non-woven 25 gsm
- Middle layer: Melt blown 25gsm
- Inner layer PP Non-woven 25gsm
- Size 175mm x 95mm

Registered Office:
AIDAPT BATHROOMS LTD.
LANCOTS LANE, SUTTON OAK, ST. HELENS, WA9 3EX