



VA126SB Gel Comfort Cushion



Usage and Maintenance Instructions

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

INTRODUCTION

Thank you for deciding to purchase the Gel Comfort Cushion from Aidapt. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the item.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

- The Gel Comfort Cushion can provide extra support and comfort in cars and homes.
- Ensure that the Gel Comfort Cushion is pushed back to the rear of the seat before use.
- Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or foam filling.
- Not recommended for use outdoors.

GEL POUCH SAFETY INFORMATION

The cushion includes a gel pouch and the following first aid measures apply;

- In case of inhalation access fresh air and consult a doctor if symptoms persist.
- In case of contact with skin wash hands and affected area immediately with soap and water.
- In case of contact with eyes rinse affected eye for several minutes under running water.
If symptoms persist consult a doctor.
- In case of swallowing rinse mouth with water immediately. If symptoms persist consult a doctor.

For further details see web site www.aidapt.com for full material safety data sheet.

CLEANING

Clean your Gel Comfort Cushion using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not soak the cushion filling. The removable cover can be washed on a gentle cycle in a washing machine. After cleaning, leave the cushion to air/dry in a warm, dry place, but not in direct contact with a heat source.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

IMPORTANT INFORMATION

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk