



**VS216CL**  
**Commode Liners**  
**with Super Absorbent Pads**



**Fixing and Maintenance Instructions**

**This file is available to view and download as a PDF at [www.aidapt.co.uk](http://www.aidapt.co.uk). Sight impaired customers can use a free PDF Reader (such as [adobe.com/reader](http://adobe.com/reader)) to zoom in and increase the text size for improved readability.**

**Please read the instruction manual thoroughly for correct usage and keep a copy available for future reference.**

The Aidapt commode liner with absorbent pads are made of a leak-resistant medical grade material and include a unique super absorbent pad which turns up to 16oz of body fluids into a gel in a matter of seconds. The commode liner fits over most commode pails. These single use commode liner with absorbent pads help to eliminate the presence of stool and urine odors. They also include attached straps that you pull to close and then tie shut for efficient clean-ups. They also have biodegradable technology making them safe for the environment. The Aidapt commode liner with absorbent pad are available in a convenient 50-count box.

## **INSTRUCTIONS FOR USE**



1. Fit the liner into the commode pail and pull the strings to secure it from the bottom.



2. Place the super-absorbent pad at the center of the commode pail. After use, bodily fluids turn to gel preventing splashes and spills. Absorbs up to 450ml/ 16 fl oz of bodily fluids or 1000ml of distilled water



3. After use, pull up the drawstrings together and tie a knot around the opening to seal the liner.



4. After use, throw away the liner in the waste bin.

5. Dispose of waste properly. Do not flush the absorbent pad in the toilet to prevent clogging the system.

## DIMENSIONS

Bag .....540x400mm  
Pad .....240x130mm

## COLOUR

Bag ..... White with Blue Drawstring  
Pad ..... White

## BENEFITS

1. Helps protect staff and patients
2. Decreases contact with bodily waste and traps unwanted odors
3. Prevents spills and splashes
4. Reduces cleaning chores
5. Saves time
6. Reduces stress
7. Biodegradable
8. Non-sterile

## WARNING

**Plastic bag causes suffocation, keep away from children. Dispose of properly, or in case of infection, in biohazard waste.**

**Keep away from any sources of flame/extreme heat - will burn if exposed to flame.**

**The liner has a long shelf life provided the bag is in a good cool storage condition and not opened from the box as the EPI (biodegradable agent) needs the oxygen, UV light and heat to breakdown.**

**After use, the plastic degrades when exposed to oxygen, heat and/or sunlight, this process takes approximately 180 days.**

## TECHNICAL SPECIFICATION

Commode liner ..... LDPE (drawstring HDPE)  
Absorbent Pad ..... Super Absorbent Polymer (SAP), tissue, virgin wood pulp  
..... Absorbency 450ml 0.9% saline 35°C or 1000ml distilled water

## IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.com](http://www.aidapt.com)**  
**Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**