



## **Anti-fall Barrier PU Foam**



**VG888**

## **Fixing and Maintenance Instructions**

**INTRODUCTION**

Thank you for purchasing the Anti-fall Barrier PU Foam.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

**BEFORE USE**

Remove from packaging, unroll and allow the foam to return to its full shape. This may take 2 hours.

Please note all new materials have their own particular smell, which will gradually disappear.

**ASSEMBLY AND OPERATING INSTRUCTIONS**

Place the barrier as required on the edge of your mattress under deep fitted sheets. The anti skid silicone beaded material ensures the barrier stays in place.



1. Water resistant cover with silicone beaded fabric.
2. Must be used with deep fitted sheets.
3. Suitable for use on all bed sizes - Single, Double, King, Super King, Queen.
4. Simple to fit between the mattress and fitted sheet.
5. Removable cover is machine washable up to 60°C
6. DO NOT use bleach to clean the product, drip dry and do not iron.

**WARNING:**

**It is recommended that a risk assessment be undertaken as to the suitability of the barrier for each individual user or if used with any bed rails or other equipment.**

**TECHNICAL INFORMATION**

Length = 1344mm

Height = 120mm

Width = 190mm

Net weight = 1.2kg (2.7lbs)

**IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

A copy of the instructions leaflet can be downloaded from our website which can be enlarged if required.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.com](http://www.aidapt.com)**  
**Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**