



Broadway Rise and Recliner Chair Single and Dual Motor



VG706SA/VG706DA
Fixing and Maintenance Instructions
This is not a medical device.



INTRODUCTION

Thank you for purchasing the Aidapt Rise and Recliner Chair. The Broadway Rise and Recliner Chair is designed to assist people easily from the chair using electrically operated motors.

Please read the instruction manual thoroughly for the correct usage of the chair, especially the safety information. Ensure you keep this manual readily available for future reference after you have read it through.

SAFETY INFORMATION

WARNING: The chair must be assembled correctly according to the assembly instructions enclosed. The manufacturers of the chair, their agents and distributors are not liable for any damages caused by improper assembly of the chair.

WARNING: Please ensure you take care when adjusting chair positions to avoid trapping or pinching of body parts.

WARNING: Please lower the chair with caution as there is a risk of entrapment. Ensure there are no children or pets underneath the chair when lowering. Also, carers should stand clear of the chair when lowering.

WARNING: This chair is not suitable for use by children. Any children or animals within the vicinity must be always supervised by an adult and should never be allowed to play with the chair.

WARNING: Do not sit on footrest.

WARNING: Exercise care when lowering into the chair as shock loads can severely damage the chair and the mechanism.

WARNING: This is an electrical item and should be treated with caution. When the chair is not in use it should be returned to the seated position.

Motion Lockout Control = Remote Control

The remote control is easily removed and locks the chair in position. The position cannot be moved without re-plugging in the remote control.

- Ensure the chair is placed on a flat, level floor.
- Do not use your chair in an excessively humid or dusty environment, as it may cause electrical shock or mechanical problems. The chair must be kept in a dry room.
- Do not use your chair in room temperatures exceeding 40 degrees.
- Do not expose your chair to heaters, stoves or direct sunlight to protect the upholstery.
- Do not use your chair outdoors.
- Ensure the voltage corresponds with the specifications indicated for the chair.
- When the chair is not in use for long periods of time, disconnect the power cord from the main supply.
- Do not sit, rest or place any heavy objects on the armrest, footrest or back of the chair.
- Do not jump or play on the chair, as this is not a toy.
- To prevent any unauthorised use, Aidapt Bathrooms Ltd recommends that the plug is removed from the socket when the product is not in use.
- Do not disconnect the plug from the electrical outlet roughly or unplug by pulling the cord.
- Do not connect or disconnect the plug from the electrical outlet with wet hands. It may result in electrical shock or mechanical breakdown.
- Do not place the AC/DC adaptor cord under the chair as it may cause damage to the cord and result in electrical shock.
- Do not let the AC/DC adaptor cord hang down from the chair.
- Do not use the chair if the AC/DC cord or plug is damaged, or if the electrical outlet is excessively loose. Contact a professional for inspection.
- If necessary, the power cable may be replaced with an identical one, but it must only be replaced by a qualified technician.
- Do not touch the surface of the plug immediately after use, as it can be hot.
- Do not operate the remote control with wet hands, sharp or heavy objects.
- Avoid dropping the remote control.
- Do not step or place heavy objects on the remote-control wire.
- **Do not put your fingers and hands in between the seat and the footrest when the chair is reclining or when the chair is returning from the recline position to sitting position. You may hurt your fingers/hands badly.**
- This chair must be used in accordance with the instructions provided.
- The chair is not for professional use.
- The manufacturers of this chair, their agents and distributors are not liable for any damages to person(s) or property(s) resulting from improper use of the chair. Use caution when using the lift feature if the chair back is reclined. The back should be in the upright position when entering and leaving the chair unless a side transfer is being done using a vertical lift.

CAUTION: A sudden change in temperature from cold to warm can cause condensation on electrical and mechanical parts. It is recommended that the chair be allowed to warm up for a minimum of an hour before using to ensure proper operation and reduce the chances of mechanical breakdown.

Do not use the lift chair if any of the following events occur:

- Water is accidentally spilled on the chair. This may result in electric shock.
- The upholstery is torn, and the inside mechanism is exposed.
- You detect any malfunction or another abnormal condition during operation.

Use Caution if there is a power failure. Injury may occur when power is restored unexpectedly if the operation buttons are depressed. Avoid using the chair during severe lightning storms.

Please switch the power OFF immediately and disconnect the AC power cord from the electrical

SPECIFICATIONS

WEIGHT

The maximum user weight capacity is 135kg (21 stones)

Power requirements – 100~240V 50Hz 70W

Note motor operation: 2 minutes is available for continual use, allow 15 to 30 seconds between cycling the motor.

ASSEMBLY

Some models of chairs are packaged in two pieces consisting of the back and the base to make transport and delivery easier. Assemble the chair as per the assembly instructions below.

DIRECTIONS

- 1) Remove seat (fig. 1) and back (fig. 2) from packing bags.
- 2) Hold back the upholstery flap with both hands and line up brackets. (fig. 3)
- 3) Apply downward pressure (fig.4) and ensure both sides snap into locked position. (fig. 5)


Fig. 1



Fig. 2



DIRECTIONS CONTINUED

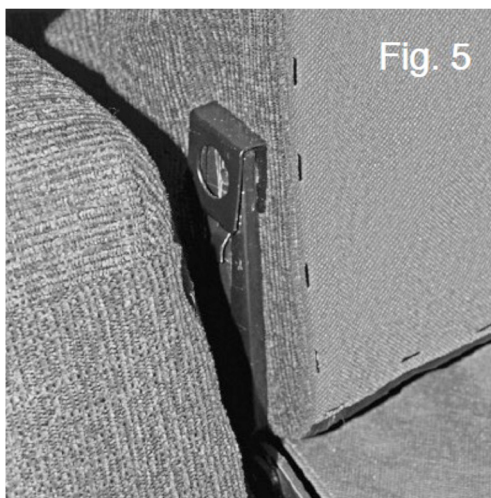

ASSEMBLY



Note: Both back receivers need to be lowered onto the flat bars at the same time and evenly to prevent binding.

Note: Do not assemble the chair if the upholstery or any of the parts of the chairs are damaged. Check all parts carefully before assembling the chair. If any problem is apparent contact the vendor.


ELECTRICAL CONNECTIONS

Note: Make sure all the wires are properly connected to ensure proper function.

1. Connector for the motor and wiring (fig. 6)
2. Connector for wiring and adaptor (fig.7)
3. Connector for adaptor wiring and plug wiring (fig. 8)
4. Hand control connection (fig. 9)

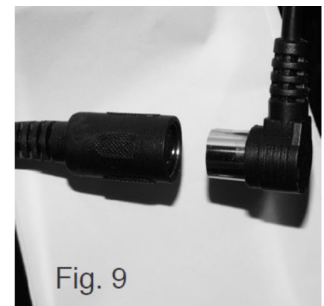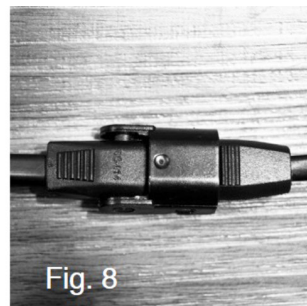

CHAIR PLACEMENT

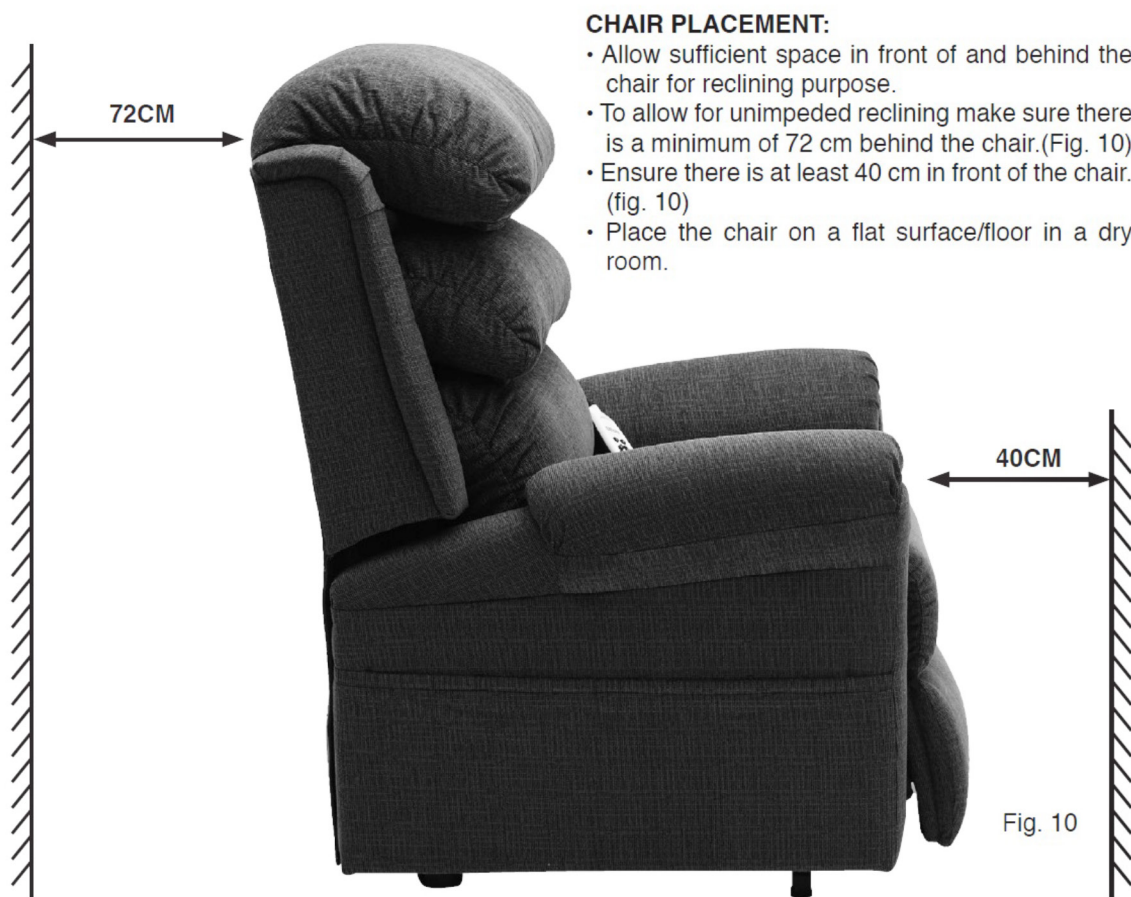

- Allow sufficient space in front of and behind the chair for reclining purpose.
- To allow for unimpeded reclining make sure there is a minimum of 65 cm behind the chair.
- Ensure there is at least 40 cm in front of the chair.
- Place the chair on a flat surface/floor in a dry room.

CAUTION: Make sure there is no obstruction behind or in front of the chair for a safe recline and lift during operation. The area between the seat and the footrest should be kept clear to avoid a pinching hazard while the chair is operating.

PRODUCT CARE & STORAGE

1. How to store

- Wipe the dust and dirt off. Keep the chair clean and away from places of high humidity.
- If the chair is not in use for a long period of time, cover it with a piece of clean cloth to protect it from dust. Store in a dry room.
- Do not store the chair under direct sunlight or in places of high temperatures. This may result in the fading of fabric colour.

2. Self-maintenance

- Before cleaning, ensure that the Main Switch is turned off and AC plug is removed for the electrical outlet.

3. Main Body

- Very mild soap can be used to clean the plastic, pipe and upholstery of the chair. Carefully wipe away all traces of the soap with dry cloth.
- To clean a dry spot, dip a piece of soft cloth into some warm soap water, wring the cloth dry, and apply to the spot.
- Do not use cleaning agents like benzene or thinner, or any other cleaners which may result in fading of fabric colour and scratching of upholstery.
- Do not remove the upholstery for washing. Ironing of upholstery is strictly prohibited.

BATTERY BACK UP SYSTEM

These chairs are supplied with a battery back-up system to allow exiting the chair during a power failure. Two 9-volt batteries are required and not supplied with the chair.

In the event of a power failure the chair will automatically switch over to battery power but only run for one cycle before the batteries are discharged. It is recommended that the batteries are replaced after each emergency use and checked periodically.



ADJUSTING THE CHAIR

This chair has a range of electrically operated adjustments that can be applied using the remote-control handset, specifically:



Seat Lift
Position



Recline
Position



ADJUSTING THE CHAIR CONTINUED



Standard Sit
Position

WARNING: Please ensure you take care when adjusting chair positions to avoid trapping or pinching of body parts.

Only the occupant or a reasonable person should operate the chair adjustments.

Chair should be returned to the seated position when not in use.

CAUTIONS:

- If the chair does not work properly, please contact the supplier.
- Only trained technicians should attempt repairs.
- There are no user serviceable parts inside.

ADJUSTING THE CHAIR CONTINUED

ADJUSTING THE CHAIR CONTINUED

Pushing this button will raise the backrest, retract the footrest and raise the chair, enabling the user to stand. The movement can be stopped by releasing the button.

Pushing this button will recline the backrest, raise the footrest and lower the chair, keeping the button fully depressed will put the chair into sleeping/relax mode. The movement can be stopped by releasing the button.



WARNING

- Ensure you do not overstretch the handset cable and that the remote controller unit is not left on the floor. When the handset is not in use, this should be stored in the pocket located on the side of the chair.
- Use only dry cloth for cleaning of the unit. Excess moisture may damage the controller.
- Avoid damaging the handset as this will result in loss of operation of the chair.
- Please ensure the chair is fully cleaned and disinfected before re-use to prevent the risk of transmitting infections if being used in a healthcare setting.
- Do not sit on the leg rest. It cannot support weight of an adult. The leg rest is folded away when being used to aid standing.

CAUTION: Please lower the chair with caution as there is a risk of entrapment. Ensure there are no children or pets underneath the chair when lowering. Also, carers should stand clear of the chair when lowering.



RE-USE

Please ensure if this chair is being re-used by another user other than the person originally using it, the new user must ensure the chair is suitable for his/her weight limit.

DISASSEMBLY

To detach the backrest from the chair both locking pins need to be released at the same time so we would suggest 2 able bodied people.


- Insert a flat head screwdriver or other flat instrument (do not use a knife or sharp object) underneath the locking pins. (fig. 11 & fig. 12)
- Apply steady outward pressure (fig. 12) to the pins on either side and apply gentle upwards pressure to the backrest (fig. 13 & fig. 14). This will release the locking mechanism.

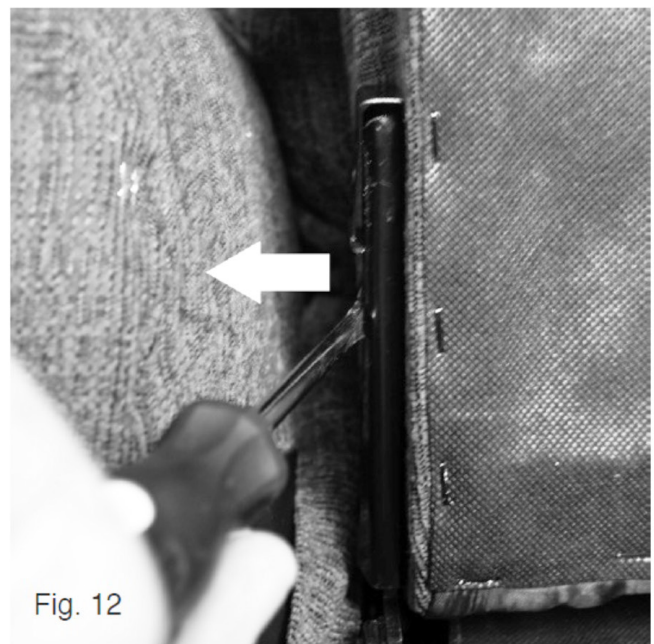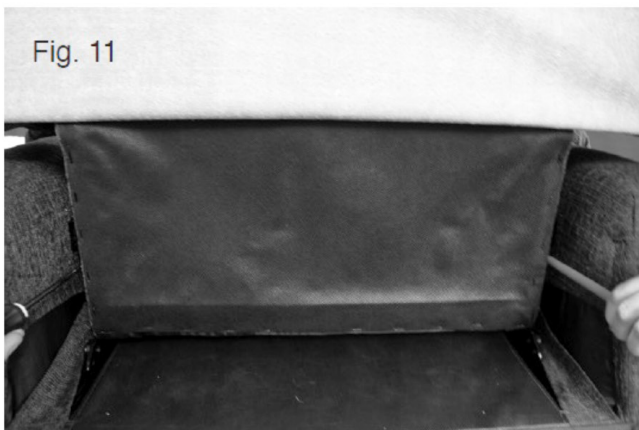

RE-USE

Please ensure if this chair is being re-used by another user other than the person originally using it, the new user must ensure the chair is suitable for his/her weight limit.

DISASSEMBLY CONTINUED

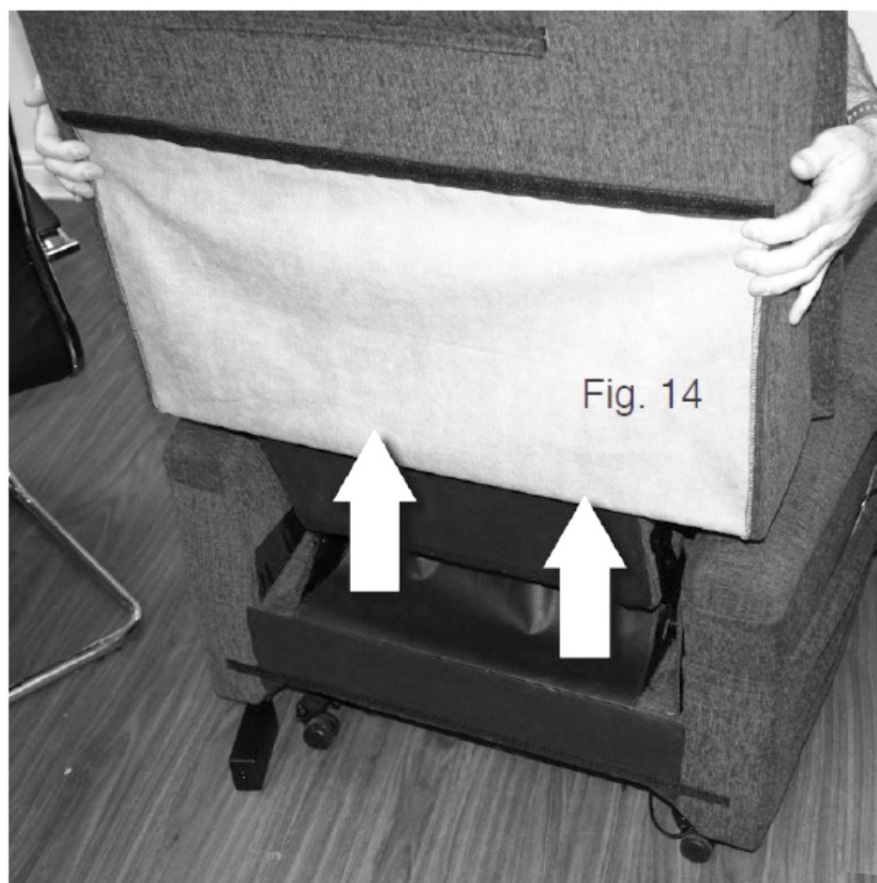

LIMITED WARRANTY INFORMATION

Aidapt Bathrooms Ltd, through its authorised distributor/service organisation, Rise and Recline chair line to the original consumer only, be free from mate workmanship problems as follows:

1. One-year warranty on chair lift mechanism
2. One-year warranty on parts and labour

WARRANTY EXEMPTIONS:

- Any alterations, misuse or abuse
- Exceeding the weight capacity (135kg/21 stones)
- Accidental damage
- Commercial use
- Battery leakage or spillage damage
- Upholstery damage, other than manufacturing defect.

Aidapt Bathrooms Ltd, at its sole discretion, reserves the right to replace or repair any part that has become defective.

The warranty does not exceed to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur. Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

IMPLIED WARRANTIES: Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

TECHNICAL SPECIFICATION

Seat Width	49cm
Seat Depth	49cm
Seat Height	49cm
Chair Width	76cm
Backrest Height	70cm
Chair Height (Sitting)	113cm
Footrest Maximum Height	55cm
Maximum User Weight	135kg (21 Stones)

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given. Please exercise common sense and do not take any unnecessary risks when using this product; as the user, you must accept liability for safety when using the product.

SERVICE WARRANTY

Aidapt Bathrooms Ltd guarantees the product free from defects in material and workmanship for a period of one year.

Should this product be operated under conditions other than those recommended, or any attempts made to service or modify the product, then the warranty is rendered void.

The product you buy may differ slightly from the illustrations. This warranty is in addition to and does not affect your statutory rights.

Our guarantee is administered by our retailers.

If your product arrives damaged, you must contact the retailer from whom you bought it. The retailer's contact details will be on the invoice that arrived with the product, or on the email, you received when you placed the order. Do not contact Aidapt Bathrooms Ltd, only your retailer can arrange a replacement or refund.

If your product fails within the guarantee period, please contact the retailer from whom you bought it.

If you have received your product and require technical help, please call our help desk on 01744 745 000. A copy of the instructions leaflet can be downloaded from our website.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.com

Email: sales@aidapt.co.uk

