



**VR157FCAL
ALUMINIUM SOLO SKANDIA TOILET SEAT
AND FRAME**



This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

Fixing and Maintenance Instructions



READ BEFORE USE

Before using your new product, you and each person that may assist you in its use should take time to read this manual and follow all instructions.

INTRODUCTION

Thank you for deciding to purchase your Commode or Toilet Seat from Aidapt. When used correctly it is designed to give many years of reliable, trouble-free service.

ALL COMMODES

Please examine your commode for any visible damage before using. If you see any damage or suspect a fault, please contact your supplier.

In this case, please do not use your product as it may compromise your safety.

VR157FCAL	Aluminium Solo Skandia Toilet Seat and Frame	Weight Limit: 150kg
VR215CAL	Aluminium Solo Skandia Toilet Seat and Frame with Lid	Weight Limit: 150kg

REPLACEMENT PARTS

VR215CAL	Lid	V5226	Replacement Sleeve
VS248	Bucket (10 Litre)	VR157CAL	Replacement Toilet Seat

Do not exceed the weight limit stated- doing so could put the user at risk.

NB. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the product for the particular user.

ADJUSTMENT OF LEGS

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the height you need, re-install the 'E' clip ensuring that it has passed through the extension leg and is fitting neatly and snugly around the commode leg. (see fig. 2)

Make sure that all legs are sitting evenly onto the floor; never extend legs in a manner that the commode is slanting. This will compromise the safety of the user.

COMMUNE BUCKET


The Skandia comes complete with a sleeve. If required a commode bucket can be used. To empty bucket simply lift the toilet seat and fix the lid provided onto the bucket to avoid splashing and carefully raise the bucket and dispose of the contents into a proper sluice or WC. Never use an open drain, sink or basin. This could cause a serious health risk. Clean out the residue with a suitable disinfectant and refit to the commode after use.

CLEANING

All commodes must be cleaned using a non-abrasive cleaner or can be pressure washed. However, always ensure that any chemicals you use with your pressure washing will not affect chrome plate, polymer coating or epoxy powder coating. After cleaning use a water dispersing agent such as WD40 on all joints and castors to reduce the chance of permanent rust or seizure. The snap on toilet seat, and if used lid, can be removed for easy cleaning.

TOILET SEAT

The Skandia comes complete with a snap on toilet seat and if required a snap on toilet lid.

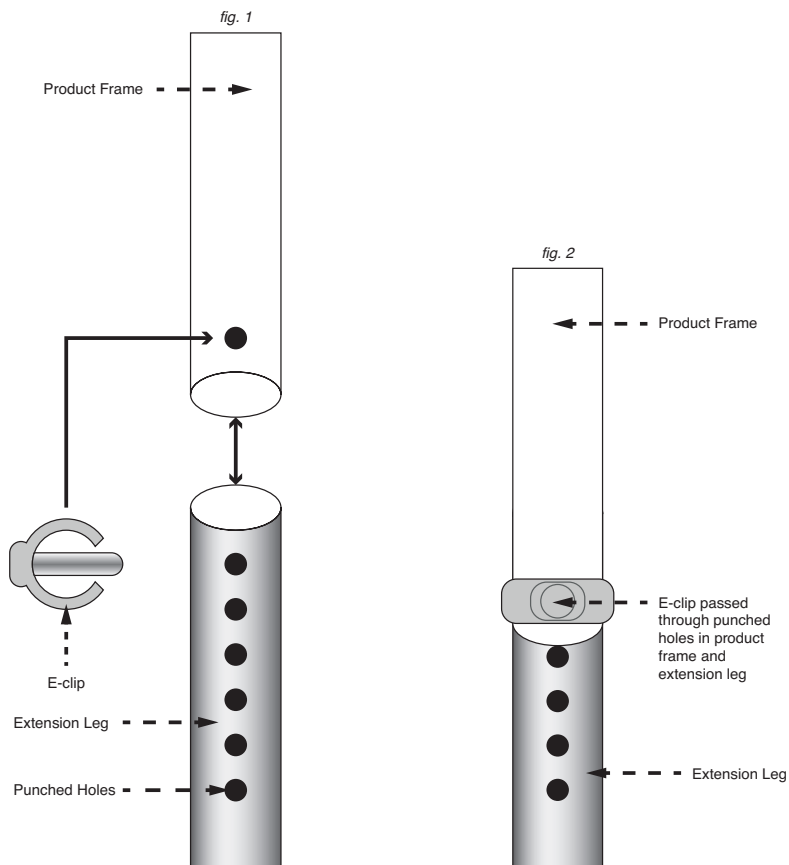

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

Do not allow children or unauthorised persons to play with the equipment or use without proper supervision.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any necessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St. Helens, WA9 3EX
 Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
 Email: sales@aidapt.co.uk

EU: Aidapt BV, Algolweg 22, 3821BH, Amersfoort, Netherlands

