



Scooper Dish with Suction Base



VM903A Single
VM903AB Twin Pack

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Scooper Dish from Aidapt. The dish is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

This useful Scooper Dish has a secure, non-slip suction base to grip a hard, flat surface such as a table, trolley or lap tray and is specially shaped with an extended lip section on one side to make it possible to scoop food into a spoon or onto a fork using one hand, assisting independent dining.

INSTRUCTIONS FOR USE


- Ensure that the surface on which you will be positioning the Scoop Dish is clean and dry.
- Position the Scoop Dish as desired then press down to secure it to the surface with the built-in non-slip suction base.
- Fill the Scoop Dish and use the specially shaped raised lip to scoop food onto a fork or spoon by pushing the utensil against and up the side of the raised lip.
- To remove the Dish break the suction by raising one edge slightly and lifting the suction base, then twist the base to fully release the suction and lift it free from the surface.



NB. If the bowl is not empty, care must be taken when removing the bowl from the surface to avoid spillage.

The suction base should be removed if the bowl is to be placed in the microwave. This can damage the suction base and also cause injury should the base stick to the microwave plate.

Net Weight: 0.26kg
Gross Weight: 0.29kg

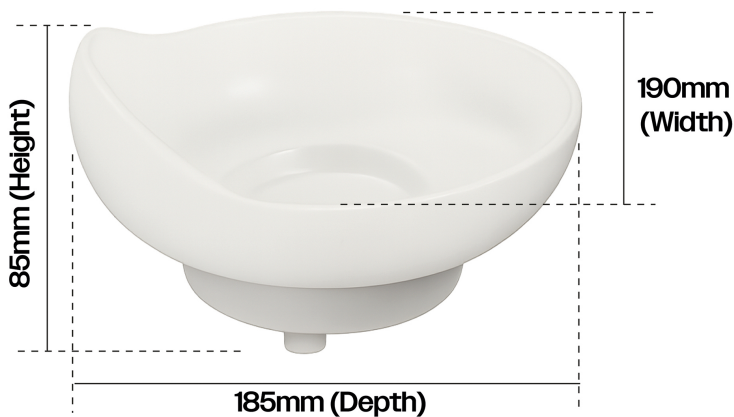

Dishwasher Safe



Microwave Safe



Easy to Clean



Food safe Plastic

Avoid Using the Suction Pad when Heating Food

CLEANING

Clean your Scoop Dish using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. CIF and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you rinse then wipe dry the equipment after cleaning.

The Scoop Bowl can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.




Aidapt Bathrooms Limited
Lancots Lane
St Helens
Merseyside, WA9 3EX UK

Tel: 01744 745020

Overseas enquiries
00441744 745020

Email: sales@aidapt.co.uk
Web: www.aidapt.co.uk

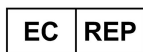

Aidapt B.V

Algolweg 22
3821BH

Amersfoort, Netherlands

Email: nederland@aidapt.eu
Web: www.aidapt.eu