



## **Cane Holder**



**VP159G Single**

**VP159GA Twin Pack**

**User Manual**

## INTRODUCTION

Thank you for deciding to purchase this Cane Holder from Aidapt.

### BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault. Do not use your product but contact your supplier for support.

### INTENDED USE

This Cane Holder is designed to enable your walking stick/cane to remain readily available when undertaking tasks that require two hands.

It allows the walking stick\cane to be secured to almost any solid horizontal surface with a lip.

### USE OF THE CANE HOLDER



#### **Step One**

*Squeeze the Cane Rest's wings gently together to open up the small circular part*



#### **Step Two**

*Attach the circular part to the shaft of your cane and release the wings*




#### **Step Three**

*The Cane Rest's wings can now be positioned against a surface and it can be left attached to your cane, it won't fall off*



Beware of finger traps when using the cane holder.

## Product Dimensions

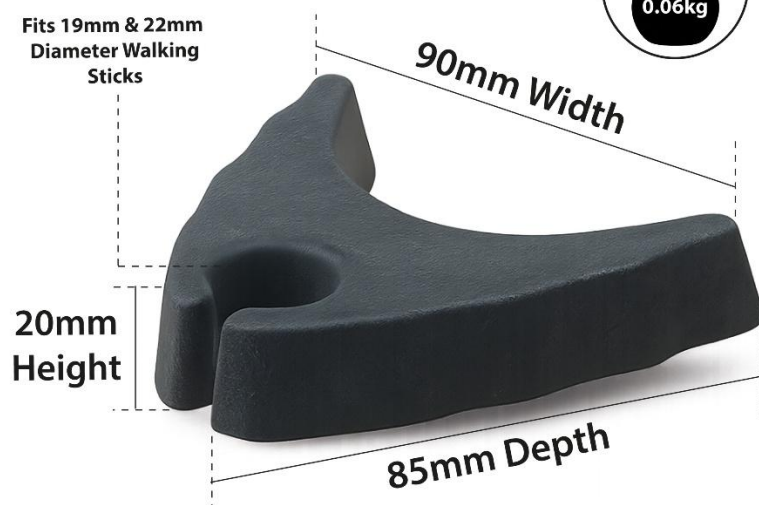

### CLEANING

Clean with a soft cloth. Abrasive cleaners and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the item after cleaning and leave to fully dry in a warm, dry place, but not in direct contact with a heat source.

### IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you any queries concerning the assembly/use of your product.




Aidapt Bathrooms Limited  
Lancots Lane  
St Helens  
Merseyside, WA9 3EX UK

Tel: 01744 745020

Overseas enquiries  
00441744 745020

Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)

Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)

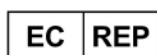

Aidapt B.V

Algolweg 22  
3821BH

Amersfoort, Netherlands

Email: [nederland@aidapt.eu](mailto:nederland@aidapt.eu)

Web: [www.aidapt.eu](http://www.aidapt.eu)