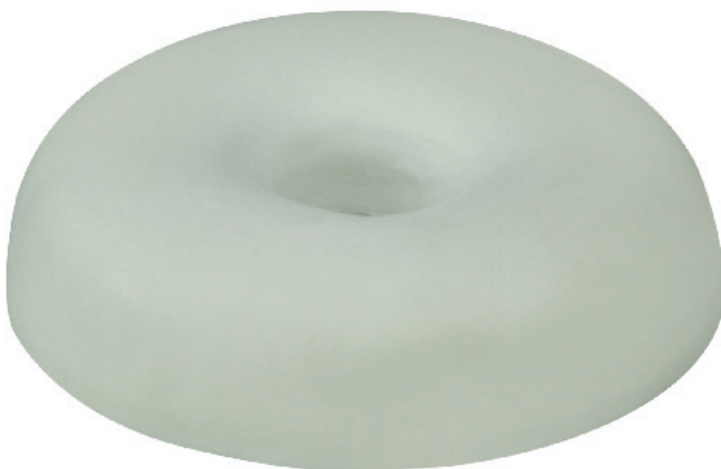
Pressure Relief Ring Cushion

VM934A

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Pressure Relief Ring Cushion from Aidapt. The Pressure Relief Cushion is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

This donut-shaped Pressure Relief Ring Cushion in memory foam reduces the pressure when sitting. It also helps to distribute weight evenly and makes it easier for the user to maintain the correct seated position. A washable cotton cover is also provided.

USING YOUR PRESSURE RELIEF CUSHION

The Pressure Relief Cushion can be used with most types of seats, including use in car seats as an infill.

NB: Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or filling.

CLEANING

Clean your Pressure Relief Cushion using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not soak the cushion foam filling.

The cover can be removed for washing on a low heat with similar colours.

After cleaning, leave the cushion and cover to dry in a warm, dry place, but not in direct contact with a heat source.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

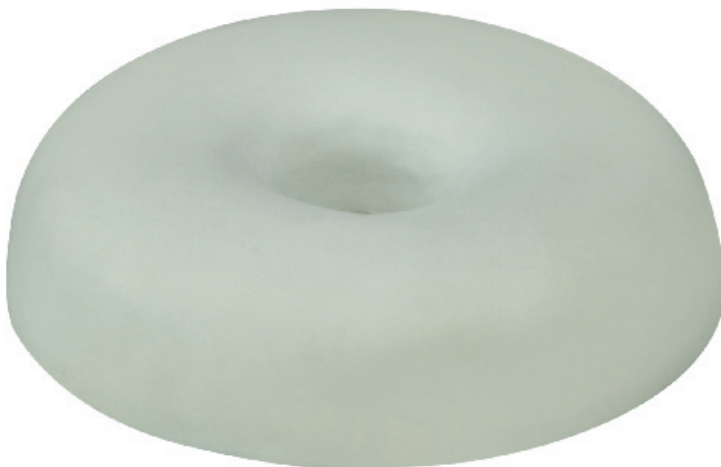
Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

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Altai Germany (Altai Europe Ltd)
Aidapt B.V, Algolweg 22, 3821BH, Amersfoort