



VR278B
Outdoor Half Step



This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Outdoor Half Step from Aidapt.

NB. A RISK ASSESSMENT SHOULD BE UNDERTAKEN AS TO THE SUITABILITY OF THE PRODUCT FOR THE PARTICULAR USAGE.

Weight Limit: 135 kg (21 stone)

Do not exceed the weight limit stated- doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage if you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

This lightweight Outdoor Half Step from Aidapt is designed to be used to help reduce heights of door thresholds and can be used both indoors and outdoors.

INSTRUCTIONS FOR USE

Position the Outdoor Half Step in a suitable location as required ensuring the surface area is level and free from any obstructions that may compromise the stability of the half step. Step up onto the grey surface area ensuring your weight is distributed centrally and step off over the threshold.

CLEANING

Clean your Outdoor Half Step using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE AND MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.com
Email: sales@aidapt.co.uk



Name: Aidapt Bathrooms Ltd.
Address: Lancots Lane, Sutton Oak, St. Helens, WA9 3EX
Tel: 01744 745 020
E-mail: sales@aidapt.co.uk



Name: Aidapt B.V.
Tel: +31 0332779210
E-mail: nederland@aidapt.eu