



## **Shower Chair**



**VB499FC**

## **Assembly and Operating Instructions**

Aidapt Bathrooms Limited  
Lancots Lane  
St Helens  
Merseyside  
WA9 3EX

Telephone: 01744 745020  
Email: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)  
Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)



## INTRODUCTION

Thank you for purchasing the height adjustable Shower Chair from Aidapt. Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier. These instructions should be read carefully and left with the user of the product for future reference.

## INTENDED USE

The height adjustable shower chair is ideal for users with reduced mobility and those who cannot stand for long periods. This product must only be used on flat, stable surfaces.

## ADJUSTMENT TO EXTENSION LEGS

Grip the extension leg with one hand whilst pushing the metal locking pin down with the other hand so the locking pin retracts fully into the shaft (see fig.1).

Select required height by adjusting the top section of the shaft in an upwards or downwards motion and release the metal locking pin in front of the required hole position (see fig.2, fig. 2(a)).

Never install the extension legs unevenly or slanting, this will compromise the safety of the user.

**Prior to use, always ensure that the locking pin fully protrudes from the selected hole position. Apply downward pressure on the handles of the shower chair to test that the shower chair has been assembled correctly.**



fig. 1



fig. 2

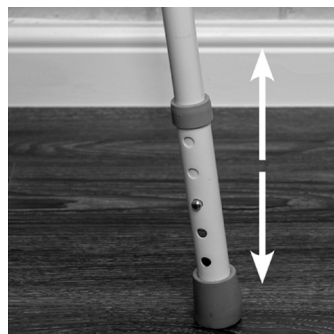


fig. 2a

## USE OF BATH OILS

Caution should be observed if you are using any form of bath oils or oily skin preparation with your shower chair as this may impair your natural skin resistance.

In the event of using any such oil, your seat must be thoroughly cleaned with a suitable detergent to break down any oily residue. Failure to observe this recommendation could impair your safety.

**CLEANING**

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chair using a non-abrasive cleaner or mild detergent with a soft cloth avoid using lime scale remover. Abrasive cleaners or abrasive cleaning pads will seriously damage your product beyond repair.

If to be disinfected by heat, one of the following three temperatures and duration of exposition can be used:

- a) a temperature of 90°C for 1 minute**
- b) a temperature of 85°C for 3 minutes**
- c) a temperature of 80°C for 10 minutes**

**Please note it is recommended that the chair is cleaned and dried off daily. This will also help to reduce the build up of lime scale on the product.**

**MAINTENANCE**

Please check the shower chair periodically for any signs of damage and if in doubt contact your supplier.

**TECHNICAL INFORMATION**

Width .....	545mm
Depth.....	470mm
Height.....	840-940mm
Net Weight .....	3.9kg
Max. User Weight.....	127kg (20 stones)

**IMPORTANT INFORMATION**

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

## SERVICE WARRANTY

Aidapt Bathrooms Ltd guarantees the product free from defects in material and workmanship for a period of one year.

Should this product be operated under conditions other than those recommended, or any attempts made to service or modify the product, then the warranty is rendered void.

The product you buy may differ slightly from illustrations. This warranty is in addition to, and does not affect your statutory rights.

Our guarantee is administered by our retailers.

If your product arrives damaged, you must contact the retailer from whom you bought it. The retailers contact details will be on the invoice that arrived with the product, or on the email you received when you placed the order. Do not contact Aidapt Bathrooms Ltd, only your retailer can arrange a replacement or refund.


If your product fails within the guarantee period, please contact the retailer from whom you bought it.

If you have received your product and require technical help, please call our help desk on 01744 745 000.

A copy of the instructions leaflet can be downloaded from our website.



Aidapt Bathrooms Limited  
Lancots Lane St Helens  
Merseyside, WA9 3EX UK  
T: 01744 745020  
E: [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)  
W: [www.aidapt.co.uk](http://www.aidapt.co.uk)

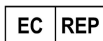

Aidapt BV Algolweg 22  
3821BH Amersfoot  
Netherlands  
E. [nederland@aidapt.eu](mailto:nederland@aidapt.eu)