

## User Instruction

### VP159D Walking Stick/Crutch Holder



The VP159D walking stick/crutch holder can be easily secured to a wheelchair, wheeled walker or walking frame, allowing a walking stick or crutch to be securely fixed to an occasional wheelchair or rollator until needed by the user. It can be secured to a 1" (26mm) tube.

#### Use

1. Pull back one of the handles, as per fig 1 Note: it does not matter which end of the holder is used.
2. Turn the handle counterclockwise until the clamp opens enough to allow the wheelchair/rollator or walking frame tube to slide into place, as per fig 2
3. Slide the clamp around the tube and turn the handle clockwise until it tightens around the tube. Ensure that it is tightened enough that it does not slide up/down the tube. Fold the handle back over the clamp, as per fig 3.
4. Repeat on the second clamp to secure the walking stick or crutch

 **Beware of finger traps. Do not use if damaged.**

