



This user manual contains essential information in relation to safe use and should be retained with the chair for reference

Rise & Recline Chairs

The Billingham



VG710



VG710W

User Manual

VG710 Rise & Recline Chair (dual motor)

VG710W Rise & Recline Wall Hugger Chair (dual motor)

Max Load 160kg (25 stone)



Aidapt Bathrooms Limited
Lancots Lane
Sutton Oak
St Helens
Merseyside WA9 3EX UK



1.0 Introduction

Featuring a powerful Okin dual motor system, these chairs allow independent control of the backrest and footrest, enabling seamless transitions between three key positions: lifting, sitting, and sleeping. Whether you need to gently lift to a standing position or recline fully for rest, the easy-to-use handheld remote control ensures smooth operation with clearly labeled buttons. VG710W wall hugger requires a clearance of 14cm from the wall.



2.0 Safety Information

IMPORTANT INFORMATION

- **This instruction manual forms an integral part of the product and must be kept with the chair at all times.**
- If the product is transferred, sold or moved to a new user, this manual must remain with the product.
- Before use, inspect the product and the mains lead for any signs of damage.
If damage is found, do not use the product and contact your supplier immediately.

GENERAL WARNINGS

- The chair must be assembled correctly in accordance with the instructions provided. Failure to assemble the product correctly may result in injury or damage.
- **The manufacturer accepts no liability for damage or injury caused by incorrect assembly, misuse, or failure to follow these safety instructions.**
- This product is an electrical appliance and must be used in accordance with these safety requirements.
- **Maximum user weight: 160 kg (25 st). Do not exceed this limit.**

USE BY CHILDREN AND VULNERABLE PERSONS

- This appliance may be used by children aged **14 years and above**, and by persons with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, **only if they have been given supervision or instruction concerning safe use of the appliance and understand the hazards involved.**
- Children shall not play with the appliance.
- Cleaning and user maintenance shall not be carried out by children without supervision.
- The product must not be left unattended during operation.

ELECTRICAL SAFETY

To reduce the risk of electric shock, fire or injury:

- This appliance is for **indoor use only**.
- Do not use this product in bathrooms, shower rooms, swimming pool areas or other wet or humid environments.
- Ensure the mains supply voltage corresponds with the rating label on the product.
- Do not use the appliance if the mains lead or plug is damaged, or if the socket outlet is loose.
- Do not unplug the appliance by pulling on the mains lead.
- Always ensure the appliance is switched off and unplugged from the mains supply when not in use.
- Do not route the mains lead beneath carpets, rugs or floor coverings.
- Do not allow the mains lead to hang freely where it may cause a trip hazard.

ELECTRICAL SAFETY (continued)

- Do not place the adaptor cable beneath or under the chair, as damage may occur.
 - Do not cover the appliance.
-

ENVIRONMENTAL AND LOCATION WARNINGS

- Place the chair on a firm, flat and level surface.
 - Do not use outdoors.
 - Do not use in environments with high concentrations of dust, fibres or airborne particles.
 - Do not use in areas where flammable liquids or gases (e.g. petrol, solvents, paints) are used or stored.
 - Keep the chair away from heaters, stoves and direct sunlight to prevent overheating or damage to upholstery.
-

OPERATIONAL SAFETY WARNINGS

- Take care when adjusting the chair to avoid trapping or pinching body parts.
 - Keep hands, fingers and loose clothing away from moving parts, particularly between the seat and footrest.
 - Lower the chair with caution. Ensure no children, pets or objects are beneath the chair during operation.
 - Do not sit or stand on the footrest.
 - Do not drop heavily into the chair, as shock loads may damage the mechanism.
 - Do not climb, jump or play on the chair. This product is not a toy.
 - Do not sit, lean or place heavy objects on the armrests, footrest or backrest.
-

LIFT FUNCTION AND MOVEMENT

- Use caution when operating the lift function if the backrest is reclined.
 - The backrest should be upright when entering or leaving the chair, unless a side transfer is being carried out.
 - Ensure the castors are braked at all times.
 - **This chair is not intended for transporting or transferring persons and must not be moved whilst occupied.**
-

TEMPERATURE WARNING

- Sudden changes in temperature may cause condensation on electrical or mechanical components.
 - Allow the chair to acclimatise for at least **one hour** before use after moving it from a cold to a warm environment.
-

DO NOT USE THE PRODUCT IF:

- Water or liquid has been spilled onto the chair.
 - The upholstery is torn and internal mechanisms are exposed.
 - Any abnormal noise, malfunction or unusual operation is detected.
 - The product or mains lead shows visible damage.
-

SERVICE AND MAINTENANCE

- This product contains no user-serviceable parts.
- Repairs must only be carried out by qualified service personnel.
- Unauthorised repairs or modifications will invalidate the warranty and may create safety hazards.

3.0 ASSEMBLY

 **CAUTION:** Due to the weight of the product 2 person assembly is strongly recommended.



Assemble the chair as per the assembly instructions below.

DIRECTIONS

1

Insert backrest into seat.


Ensuring the two metal inserts on the seat slide into the clips on the backrest with a click.

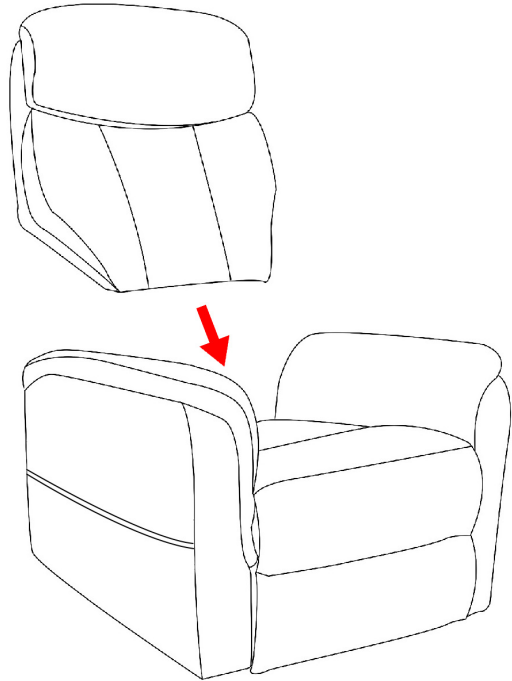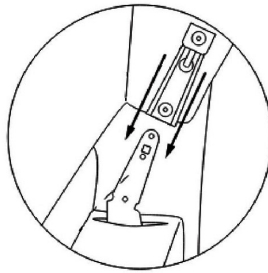

2

Lower the backrest onto the seat base prongs, then firmly push downwards.

An audible "click" will confirm the backrest is secure.

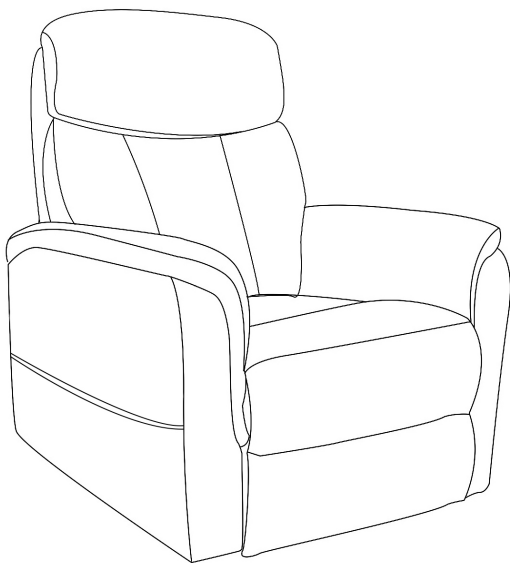


3

Lift the backrest flap to reveal the chair's wiring, then connect the power cables together.




4

Move the chair into a suitable position with plenty of space to rise and recline.

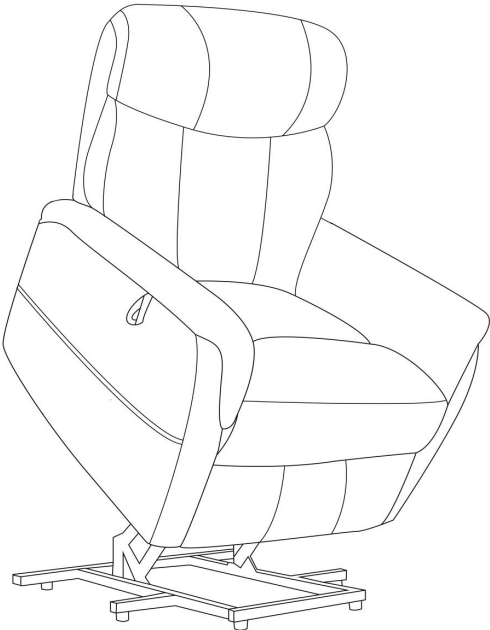


4.0 USING THE HAND CONTROL

4.1 VG710 Rise & Recline

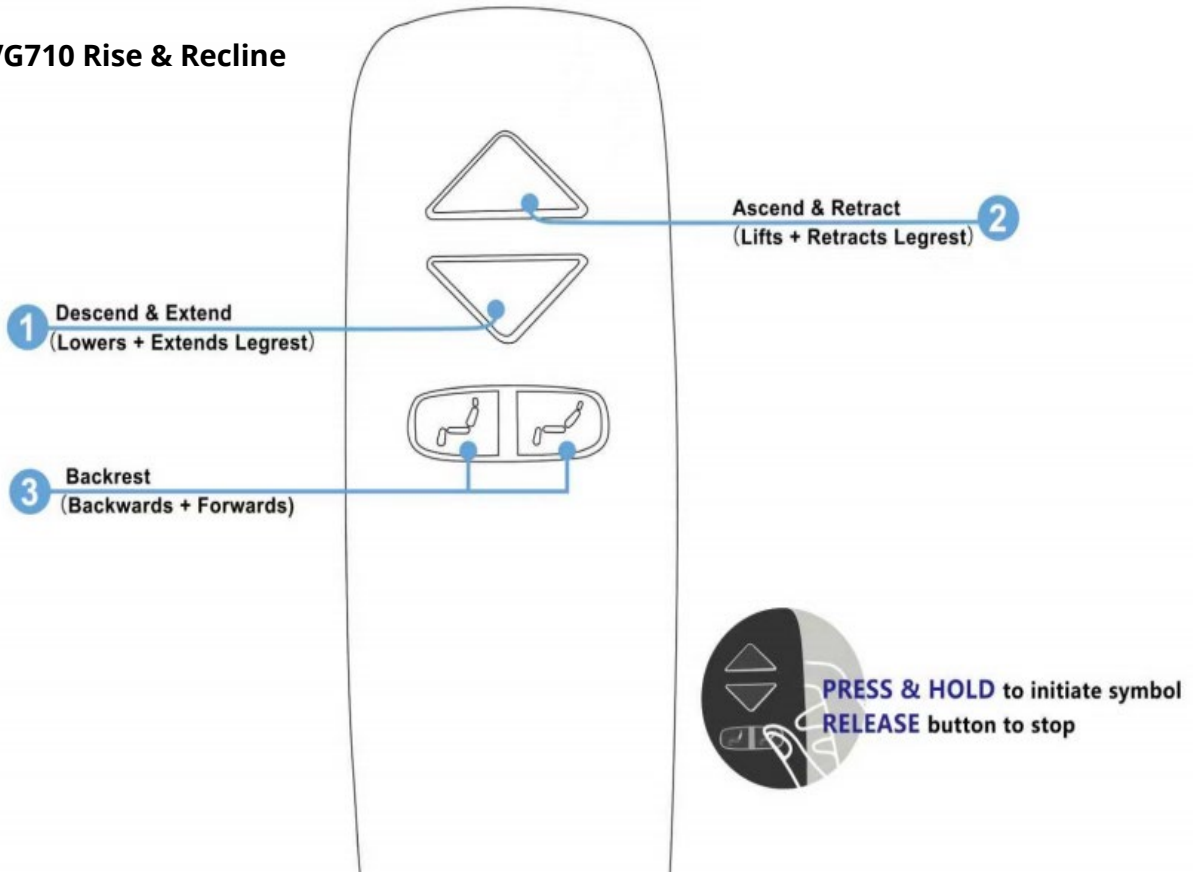

4.2 VG710W Wall Hugger

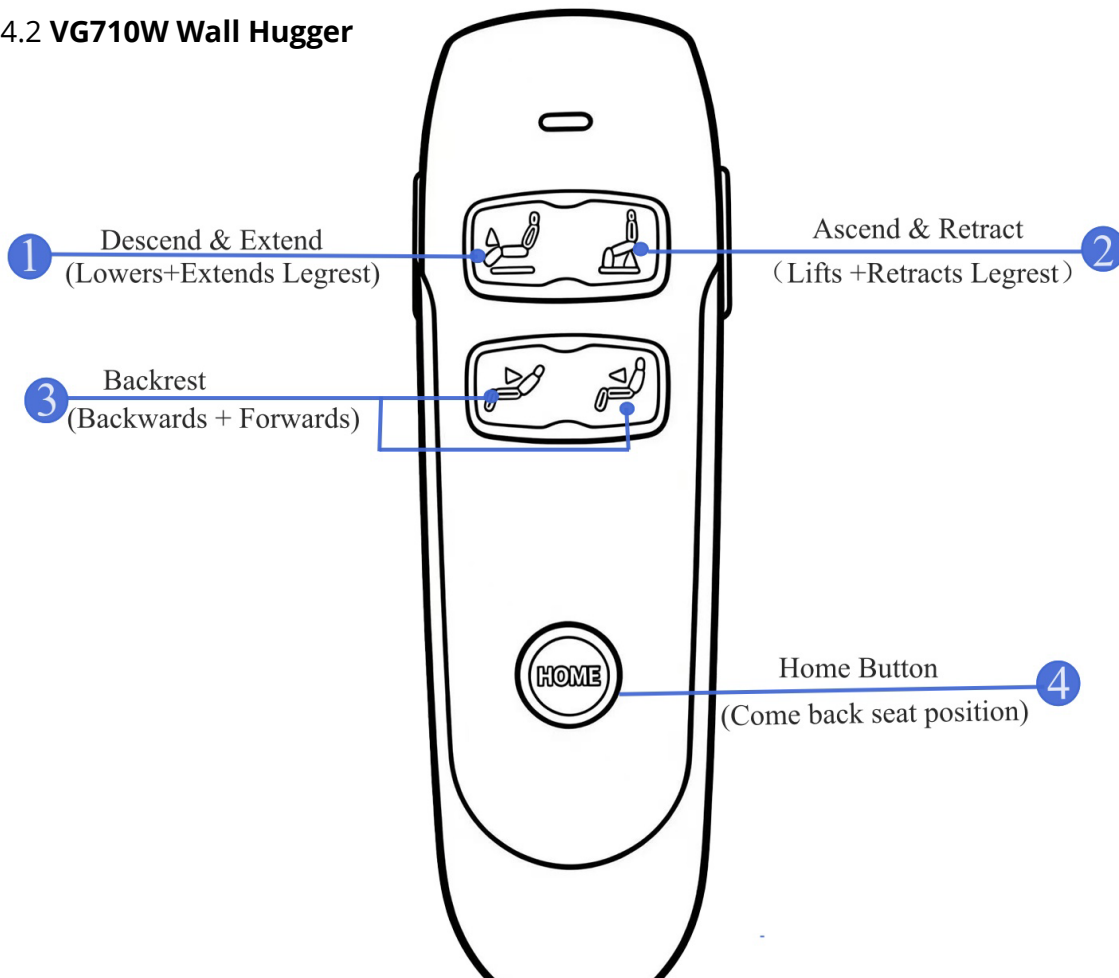

5.0 CONNECTING THE POWER

Your chair needs to be connected to a standard UK electrical socket.

1. Connect the power supply to the chair power lead, fig 1.
2. Plug in the power supply using a standard UK domestic plug.



Fig 1.



5.1 EMERGENCY POWER SUPPLY

The chair includes an emergency device in case of power failure.

The emergency power supply requires 2x 9V batteries.

**When there is no power,
please use battery back up
to reset the chair position.**

Use 9 volt batteries (not supplied)



Open the cover



Connect the batteries



Put the batteries in
Close the cover




Check the light is on (green)
before operating

6.0 USING THE RECLINER

Position the chair on a level surface with enough space to rise and recline.



To avoid risk of injury and product damage, it is recommended that two people manoeuvre the chair into position.

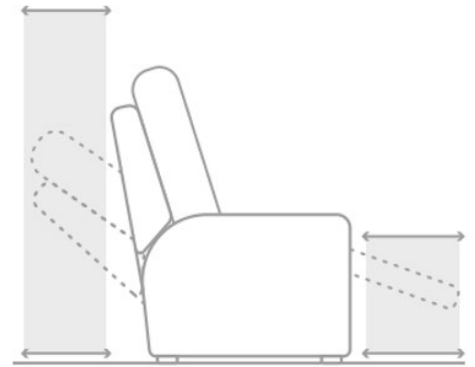

6.1 SITTING

Ensure that the backrest is in the upright position and the footrest is fully lowered. Gradually move backwards into a seated position using the armrests for support.

6.2 STANDING

Place feet flat on the floor and use the chair's lift function (see section 4 for handset controls) to gradually rise. Use the armrests for support, if needed, and ensure that you are stable before leaving the chair.

6.3 ADJUSTING THE CHAIR ANGLE

The handset can be used to adjust the angle of the chair when seated.



Ensure that pets and children are kept clear of all moving parts when operating the rise and recline chair.

7.0 CLEANING AND MAINTENANCE INSTRUCTIONS

To preserve the softness, colour, and structure of your fabric, please follow these care guidelines:

Daily & Weekly Maintenance

1. Vacuum Regularly: Use a vacuum with a soft brush attachment to remove dust and prevent dirt build-up in the fibres. Vacuum gently in the direction of the fabric's nap.

Spot Cleaning

1. Blot Spills Immediately: If liquid spills occur, blot gently with a clean, dry, and absorbent cloth. Do not rub — this can damage the fibres.
2. Mild Soap Solution: For stains, use a solution of lukewarm water and a small amount of mild, non-bleach detergent. Dampen a clean white cloth with the solution and gently blot the stain.
3. Rinse & Dry: After cleaning, blot with a cloth dampened with plain water to remove any soap residue. Then pat dry with a dry towel.
4. Test First: Always test any cleaning solution on a hidden area of the fabric to ensure colourfastness.

What to Avoid

1. No Harsh Cleaners: Do not use bleach, strong chemicals, or abrasive cleaning tools.
2. No Steam or Excessive Moisture: Avoid steam cleaners or soaking the fabric, as the fabric is sensitive to water damage and distortion.
3. Keep Away from Direct Sunlight: Prolonged exposure can cause fading or weakening of the fabric.

Additional Tips

1. Use Fabric Protection Spray (optional): A fabric-safe protector may help repel spills and stains—test before use.
2. Pet & Sharp Object Protection: fabric can snag easily. Keep pets' nails trimmed and avoid contact with sharp objects or jewelry.
3. Professional Cleaning: Consider a professional upholstery cleaning service once every 12–18 months for deep cleaning.

8.0 WARRANTIES

LIMITED WARRANTY INFORMATION

One-year warranty on chair lift mechanism (2 x motor system, including transformer)

WARRANTY EXEMPTIONS:

- Any alterations, misuse or abuse
- Exceeding the weight capacity (352lb/160kg)
- Accidental damage
- Commercial use
- Battery leakage or spillage damage
- Upholstery damage, other than manufacturing defect.

Aidapt Bathrooms Ltd, at its sole discretion, reserves the right to replace or repair any part that has become defective.


The warranty does not extend to consequential costs resulting from the fault or defect of the product: freight and travel costs, loss of earnings, or other expenses that one may incur.

Warranty is valid for the original purchaser only and the original receipt must be presented before any warranty options are considered. Please also be prepared to provide the date of purchase and serial number.

IMPLIED WARRANTIES:

Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to one (1) year from the date of purchase and to the extent permitted by law. Any and all implied warranties are excluded. This is the exclusive remedy. Liabilities for consequential damages under any and all warranties are excluded.

9.0 SPECIFICATIONS

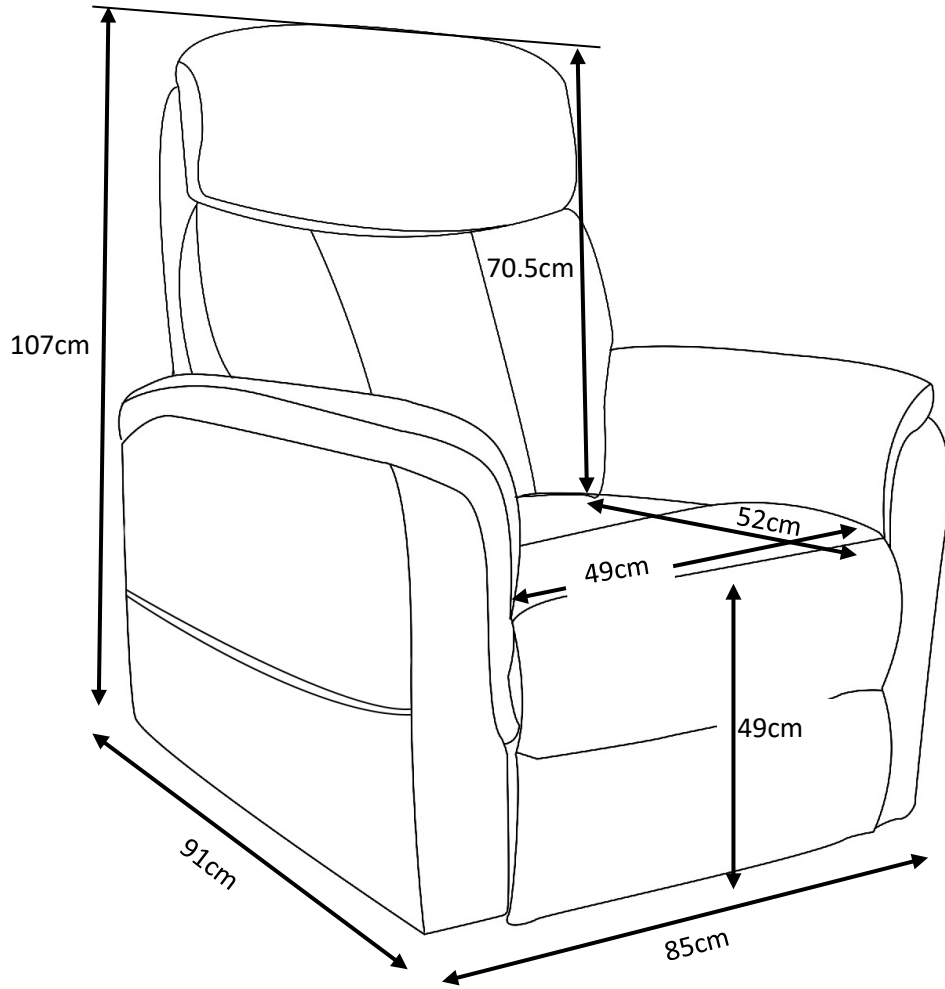

Total Height (Overall).....	107cm
Total Width (Overall).....	85cm
Total Depth (Overall).....	91cm
Seat Width.....	49 cm
Seat Depth.....	52cm
Seat Back Height.....	70.5cm
Seat Height.....	49cm
Height to top of leg when raised.....	46cm

The crossed wheelie bin symbol shown on this product indicates that the item must not be disposed of with general household waste. Proper recovery of your waste electronic and electrical equipment will ensure safety of human health and the environment.



Used batteries must be disposed of responsibly and not be disposed of with general household waste.

For more information where the product can be safely disposed of please contact your Local Authority, the dealer from where the product was purchased.




Aidapt Bathrooms Limited
Lancots Lane
St Helens
Merseyside, WA9 3EX UK

Tel: 01744 745020

Overseas enquiries
00441744 745020

Email: sales@aidapt.co.uk
Web: www.aidapt.co.uk

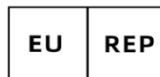

Aidapt B.V
Algolweg 22
3821BH
Amersfoort, Netherlands

Email: nederland@aidapt.eu

Web: www.aidapt.eu

